LC-6 Light Oven

SCHEU

Bedienungsanleitung
Operation Manual
Mode d'emploi
Manual de instrucciones
Istruzioni per l'uso

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LC-6 Light Oven 230 V, REF 5610.1

LC-6 Light Oven 115 V, REF 5620.1

1.) Allgemeines

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LC-6 Light Oven 115 V, REF 5620.1

1.) General

The LC-6-Light Oven is a light-polymerisation unit for light-curing resins, equipped with 2 different kind of fluorescent tubes (3 UV-A lamps, 3 blue light). The wave length range varies between approx. 350 and 450 nm, the maximum being approx. 370 nm (UV-A) and approx. 450 nm (blue light).

2.) Getting Started

Changing the light sources

For safety reasons, lamps are individually packed for transport. Pull out mains plug, pull out drawer completely, unpack the lamps and put them frontally through the openings in the mirror wall into the bulb fitting, pressing hard. The bulbs will snap in place. You should hear a click. In order to guarantee a homogeneous illumination, UVA-lamps and blue light tubes should be inserted alternately.

Dismantling the light sources:

Pull out mains plug, pull out drawer completely, let unit cool down, take hold of the lamps at the bulb and draw them out of the fitting with a quick motion. Lamps have to be changed after approx. 1.000 h, however not any later than after 2 years. Replacement lamps are available with your electronic retail market. Prior to start up, make sure to verify the mains voltage corresponds to the nominal voltage as indicated on the type plate.

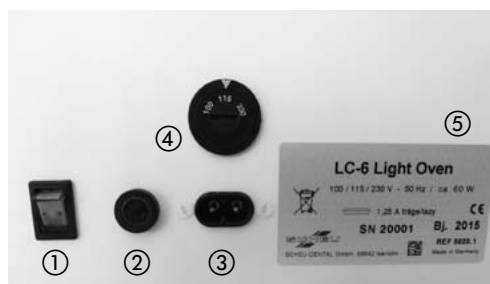
Only REF 5620.1: Adjust the right voltage on the monitoring switch on the machine back before start up.

Insert power cable in the socket at the machine back and connect it. When switching on the power switch, it flashes green. Select illumination time by pressing the button on the machine top.

Note:


In order to adjust illumination time, make sure the drawer has been pulled out for at least 2 cm. Drawer can be pulled in and out by means of the handle at the screen at the machine front.

Back

- ① Power switch
- ② Fuse
- ③ Mains socket
- ④ Voltage selector (only for REF 5620.1)
- ⑤ Type plate

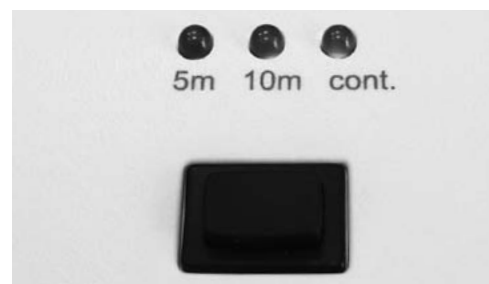Top Side

Timer button to adjust the illumination time (5 m*, 10 m*, cont.)

3.) Important Hints

Make sure not to look directly at the radiation source (risk of damaging to the eyes). Make sure not to cover the aeration openings. In the continuous mode (cont.) the machine has to be switched off after curing has been finished. A safety shutdown will run automatically after approx. 30 min.

4.) Operation

Switch on power switch on the machine back, switch flashes green.

Pull out drawer, select illumination time by pressing the timer button on the machine top, i.e. 5 m*, 10 m* or cont. The corresponding LED flashes. Place light-curing material on the drawer and slide drawer back as far as it will go. The illumination process will start automatically. In case illumination times of 5 m* or 10 m* have been set, the unit will switch off automatically after the illumination process has elapsed.

In case the continuous operation mode (cont.) has been set, illumination can be switched off by pressing again the timer button. For safety reasons, after 30 min. illumination will shut down automatically.

Hints:

If the drawer has been pulled out for more than 1 cm during illumination, unit will shut down automatically (to avoid eyes damaging). The previously adjusted illumination time will be stored after the illumination process has finished. In order to start a new illumination process, the drawer has to be pulled out for at least 2 cm before being slid back as far as it will go. The illumination time can only be changed with the drawer been pulled out.

*Hint: m = minutes

5.) Maintenance, cleaning, repair

Outer surfaces can be cleansed with a moist cleansing cloth. Attention: Avoid letting liquids enter the mains socket or the timer button. The inner reflector surfaces shall only be cleansed with a dry soft cloth. Doing so, pull out mains plug, dismantle lamps and reinstall them after cleaning.

Attention: Avoid letting liquid enter the bulb fittings. Any kind of repair has to be effected by SCHEU-DENTAL or by staff authorised by SCHEU-DENTAL.

6.) Transport, storage, disposal

Transport, storage

Transport and storage up to 6 months is subject to the following conditions:

Temperature: 0°C to 50°C

Relative moisture: 10% to 60%

Apart from this, the parameters of the operating conditions apply.

Temperature: + 10°C to 30°C

Relative moisture: 20% to 60%

Store unit in closed rooms only. Protect machine from moisture and humidity. Make sure not to expose unit to strong shocks.

Disposal:

Disposal has to be effected according to the national legal prescriptions and by authorised disposal companies.

Environmental relative materials:

Device:	Material:
Housing parts of metal	aluminium, steel
Housing parts of plastic	ABS
Transformator	copper, iron,
Control gear	
Conductor board	epoxy resin

All devices and their components can be sent back to manufacturer for disposal.

Transport charges shall be borne by sender.

7.) Technical Data: REF 5610.1, REF 5620.1

Nominal voltage: REF 5610.1, 230 Volt
REF 5620.1, 100 / 115 Volt, reversible

Nominal frequency: 50/60 Hz
Power consumption: approx. 60 Watt
Fuse: 1,25 AT
Quantity of lamps: 6 pcs. of 9 Watt (UV-A or blue light)
Illumination times: 5 m*, 10 m*, continuous mode
(Safety shutdown after 30 min.).

Start: automatical start when drawer is slid back.
Endurance of lamps: approx. 1.000 hours resp. 10.000 activation operations, resp. 2 years
Inner dimensions: approx. 170 x 120 x 160 mm (WHD)
Outer dimensions: approx. 270 x 240 x 180 (LWH)
Weight: approx. 3 kgs

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SCHEU-DENTAL GmbH