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Apply one thin layer of BIOPLAST® insulating agent only to those parts of the working model which will be covered by the mouthguard. Embed model in pellets with the clear V-shaped part of the BIOPLAST® XTREME covering the desired extension of the anterior section (eg. from 3-3). Place pellet cover on pellets. Make sure the plate is fixed with the soft side down and the labels being placed in the direction of the heater. The imprint on the label has to be readable. Heat the plate by setting the code or recommended heating times according to the instructions. Lock pressure chamber and open after cooling phase has finished.

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Apply BIOPLAST® insulating agent on the antagonist model and put the models into the articulator for counter bite. Cool down with compressed air or water and remove. Polish the edges using the finishing set. If necessary, polish thermally.

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Warm up the occlusal surfaces under the infrared heater using the heating support.

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Materialliste:
BIOPLAST® XTREME 5,0 mm
BIOPLAST®-Isoliermittel
Heiz-Support

List of materials:
BIOPLAST® XTREME 5,0 mm
BIOPLAST® insulating agent
Heating Support

Liste des matériaux:
BIOPLAST® XTREME 5,0 mm
BIOPLAST® agent isolant
Support de chauffe

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Cut mouthguard to the desired shape. Polish the edges with OSAMU-Polisher or DIMO®PRO wheel.

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Innovation for safety plus.

The new Mouthguard type **BIOPLAST® XTREME** gives safety plus in sports thanks to the integrated reinforcement in the delicate anterior region. Hard and flat impacts can be absorbed in contact sports, using this innovative new material. **The innovation:** For the first time, different degrees of hardness are integrated in one pressure moulding material. The size of the anterior protection area can be varied individually thanks to the V-shaped hard insert.

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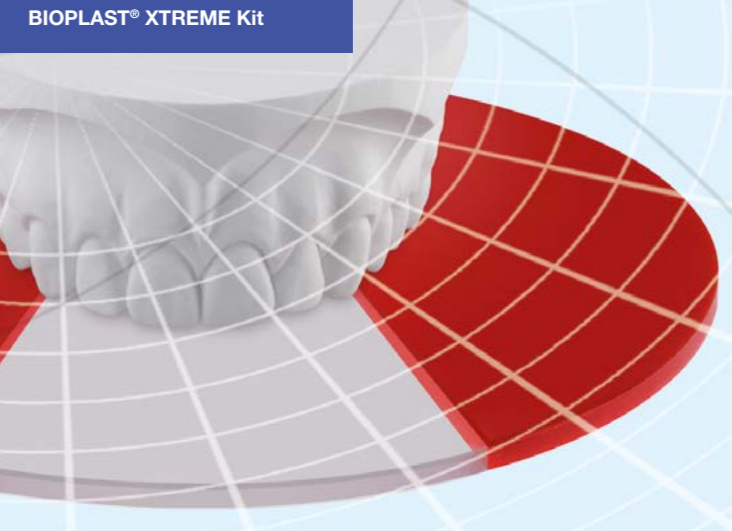
Accessories: Mouthguard Straps (REF 3319)

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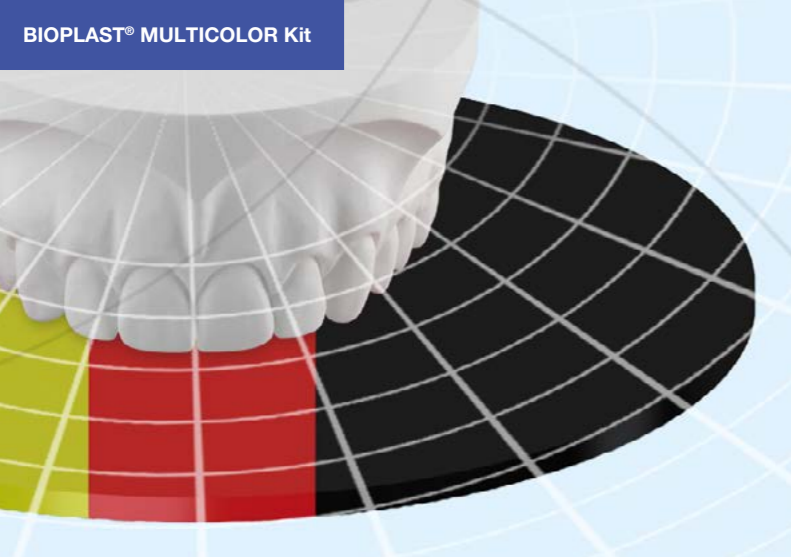
Apply a thin layer of BIOPLAST® insulating agent to the desired mouthguard area. Trim the model flat and place it on the model platform. Big models can be embedded in the pellets with the pellet cover placed over them. Heat the plate by setting the code or recommended heating time according to the instructions. Lock pressure chamber and open after cooling phase has finished.

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Cut the first BIOPLAST® with scissors or scalpel, polish the edges. Perforate it interdentially at the papilla and occlusally with a drill bitt o avoid air inclusions, the replace it on the model, clean it with lukewarm water and let dry. For optimal compound, make sure not to touch the surface again. Place model on the platform resp. into the pellets and cover with pellet cover, if necessary. Heat 2nd BIOPLAST® by setting the code or recommended heating times. Lock pressure chamber and open after cooling phase has finished. Cut material at model bottom.

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Warm up occlusal surfaces under the infrared heater and apply a thin layer of BIOPLAST® insulating agent occlusally to the antagonist model, in order to avoid plaster residues. Put the models into the articulator for counter bite. Cool down with compressed air and water and remove. Cut the mouthguard to the desired final form. Polish the edges with BIOPERM-Trimmer or with DIMO®PRO wheel. Cut the mouthguard diagonally to remove sharp edges in the labial and palatal area. Finish the edges with the OSAMU polisher at a low RPM exerting low pressure. There is no need for thermal polishing.

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Attractive colors and various designs.

Individual mouthguard is the right complement of a professional protection for your leisure time activities as well as for professional competition. For your leisure time activities the **BIOPLAST® COLOR/MULTICOLOR** is the proved and tested mouthguard, guaranteeing maximum safety.

The **BIOPLAST® COLOR/MULTICOLOR** is available in 10 attractive colors and designs. Logos and names can be inserted, if required.

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List of materials:

- BIOPLAST® MULTICOLOR 3,0 mm or
- BIOPLAST® COLOR 3,0 mm
- BIOPLAST® clear 2,0 mm
- BIOPLAST® insulating agent
- Heating support

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- BIOPLAST® MULTICOLOR 3,0 mm ou
- BIOPLAST® COLOR 3,0 mm
- BIOPLAST® clair 2,0 mm
- BIOPLAST® agent isolant
- Support de chauffe

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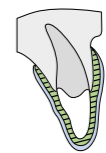
BIOPLAST® XTREME

Three Mouthguard types – meeting all requirements.

BIOPLAST® COLOR/MULTICOLOR:
 • proven mouthguard of two layers for high safety
 • for all leisure time activities

The product innovations **BIOPLAST® XTREME** and **BIOPLAST® XTREME PRO** give safety plus thanks to specific extras.

BIOPLAST® XTREME:
 • with integrated reinforcement segment in the anterior region
 • for contact sports: absorbs hard and flat impacts

BIOPLAST® XTREME PRO:
 • with integrated reinforcement segment plus hard-elastic layer in the anterior region
 • for professional contact sports where hard and flat impacts often occur, absorbing the impact power over a large surface.

All mouthguard types as described above consist at least of 2 layers, guaranteeing maximum safety and comfort.

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