// Pressure Moulding Technique

BIOPLAST® XTREME PRO Kit

Herstellanleitung / Instruction manual /
Mode d'emploi / Manual de instrucciones

SCHEU
Dental Technology

Materialliste

BIOPLAST® XTREME PRO 4,0 mm

BIOPLAST® 2,0 mm

DURAN®-Folie 0,75 mm

BIOPLAST®-Isoliermittel

Heiz-Support

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BIOPLAST® XTREME PRO 4,0 mm

BIOPLAST® 2,0 mm

DURAN® 0,75 mm

BIOPLAST® agent isolant

Support de chauffe

List of materials

BIOPLAST® XTREME PRO 4,0 mm

BIOPLAST® 2,0 mm

DURAN® 0,75 mm

BIOPLAST® insulating agent

Heating Support

Lista de materiales

BIOPLAST® XTREME PRO 4,0 mm

BIOPLAST® 2,0 mm

DURAN® 0,75 mm

BIOPLAST® líquido separador

Soporte de calentamiento

BIOPLAST® XTREME PRO Kit

Herstellanleitung	4 - 10
Instruction manual	11 - 17
Mode d'emploi	18 - 24
Manual de instrucciones	25 - 31
Heizzeiten und Codes	
Heating times and codes	
Temps de chauffe et codes	
Tiempos de calentamiento y códigos	32 - 33

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BIOPLAST® XTREME PRO

Instruction manual

Apply one thin layer of BIOPLAST® insulating agent only to those parts of the model which will be covered by the mouthguard. Trim model flat and place on model platform with the clear V-shaped part of the BIOPLAST® XTREME PRO material covering the desired extension of the anterior section (e.g. from 3-3). Make sure the side of the label with the imprint is facing the infrared heater when fixing the material. Heat the foil by setting the code or recommended heating time according to the instructions. Close pressure chamber and open after cooling phase has elapsed.

Instruction manual

CB

Cut material with scissors or scalpel on the model bottom and insulate the anterior section with a silicone spray.

For the fabrication of a protection insert: embed the model vertically in the pellets and heat DURAN® 0,75 mm by setting the code or recommended heating time. Lock pressure chamber and open after cooling phase has elapsed.

Cut the BIOPLAST® XTREME PRO to the desired extensions. Make sure the material is being cut straight edge, as cutting it diagonally might impair the optimum lamination with the 2nd BIOPLAST®. Cut the DURAN®, too, with scissors to the desired extensions.

Make sure the extension of DURAN® towards the alveolar ridge is some millimeters shorter than the extension of BIOPLAST® XTREME PRO underneath, in order to avoid problems in laminating the material.

Pierce holes into the BIOPLAST® XTREME PRO – occlusally and interdentally – by means of a bur, in order to avoid air bubbles under the protection insert and between the BIOPLAST® material.

Clean BIOPLAST® XTREME PRO with moist tissue paper to remove the silicone insulation.

Before pressurising the BIOPLAST® 2 mm, place the protection insert made of DURAN® in the front, the front of the model facing the pressure chamber.

Then heat BIOPLAST® 2,0 mm by setting the code or recommended heating time, lock the pressure chamber and open after cooling phase has elapsed. Cut material at model bottom.

Heat the occlusal surfaces under the infrared heater using the heating support with the pressure chamber closed, but not locked. Insulate occlusally the antagonist model with BIOPLAST® insulating agent, in order to avoid plaster residues. Then put models in the articulator for counter bite. Cool down with compressed air and water and remove.

Cut the material with scissors to desired extensions.

Polish the edges with BIOPERM trimmer or DIMO[®]PRO wheel. Cut the mouthguard diagonally to remove sharp edges in the labial and palatal area. Finish the rims using the OSAMU polisher at a low RPM exerting low pressure.

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SCHEU-DENTAL GmbH