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Instructions for use

Primer for fabrication of temporaries in the pressure moulding technique

SD-COMPOSITE PRIMER REF 3705.1

Area of Application

Bonding of temporaries made of DURAN® (PET-G) and acrylics for temporaries (composite).

Instructions for use

Clean pressure moulded splint of DURAN® with medical ethanol. Apply SD-COMPOSITE PRIMER to the inner surface of the DURAN® mould. Then apply composite for temporaries to the primer while it is still wet on the pressure moulded splint for compound of acrylics and splint material. Place the splint filled with composite directly in the patient's mouth or on the insulated model with the preparation. Please follow manufacturer's instructions for curing of acrylics. Proceed with finishing and polishing.

Storage

Store the material at a temperature of 10-25 °C. Shut bottle firmly after use. Keep away from sources of ignition.

Shelf life

The maximum shelf life is printed on the label of each bottle. Do not use after the expiry date.

Form of delivery

25 ml glass bottle with integrated brush

Labelling according to Regulation (EC) no. 1272/2008:

H225	Highly flammable liquid and vapour.
H315	Causes skin irritation.
H317	May cause an allergic skin reaction.
H335	May cause respiratory irritation.
P210	Keep away from heat/sparks/open flames/hot surfaces. – No smoking.
P241	Use explosion-proof electrical/ventilating/Lighting/equipment.
P303 + P361 + P353	If on skin (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.
P321	Specific treatment (see on this label).
P405	Store locked up.
P501	Dispose of contents/container in accordance with local/regional/national/ international regulations.

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Overview of Test Results (Adhesion with splints made of DURAN® [PET-G] and acrylic for temporaries):

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