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How to fabricate a TFA splint using the TFA Kit acc. to Dr. Echarri

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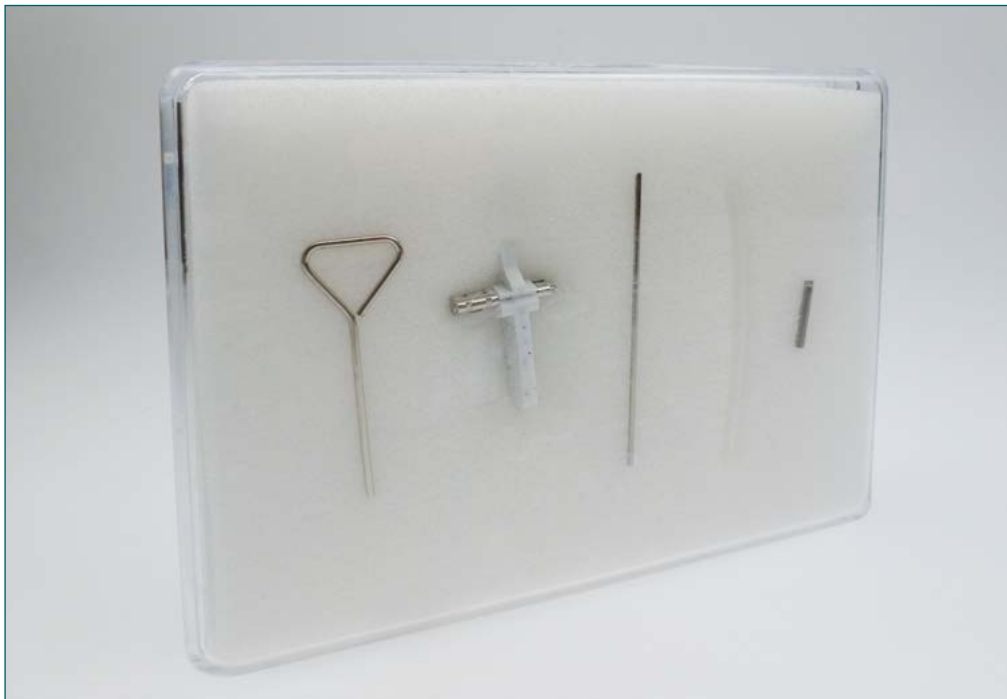
How to fabricate a TFA splint (Tractive Force Appliance) using the TFA Kit acc. to Dr. Echarri

The TFA Kit was developed by Dr. Echarri and is used for space closure of posterior teeth. The telescope sleeves with guiding rod are fixed from the labial side and the special traction screw from the palatal side.

We recommend the following specialized literature for further information:

CA® Book Volume 2 Advanced

GB: REF 5309 / German: REF 5307 / Spanish: REF 5318

Important:

Make sure not to exceed the maximal tooth movement in the CA® CLEAR ALIGNER technique acc. to Dr. P. Echarri (CA® Book volume 2, Advanced, pages 52/53).

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TFA Kit acc. to Dr. Echarri (REF 5329.1)

All components are also available separately:


- REF 5327.800 Guiding rod, 1 pc.
- REF 5323.810 Telescope sleeve, 1 pc.
- REF 5326.800 Silicone tube, 1 pc.
- REF 5329.001 Special traction screw (comes with key for expansion screws), 1 pc.

Working material:

- REF 3207 ISOFOLAN®
- REF 3342 DURASOFT® pd 1.2 mm or
- REF 3388 DURASOFT® pd 1.8 mm
- REF 3218 BIOCRYL®-RESIN monomer
- REF 3221 BIOCRYL®-RESIN polymer
- REF 3259 DURASPLINT® monomer
- REF 3258 DURASPLINT® polymer


Additionally recommended material:

- | | | | |
|----------|---------------------------|----------|-----------------------------|
| REF 3442 | SIL-KITT red | REF 3007 | Pellet cover (small) |
| REF 3220 | Blocking out putty | REF 3376 | DIMO® PRO Slim |
| REF 3460 | SD Foil scissors A | REF 3382 | DIMO® brown, coarse |
| REF 3461 | SD Foil scissors B | REF 3371 | POLYFIX |
| REF 2041 | CARAT® Flat pliers | REF 3710 | SD Polishing paste A |
| REF 3268 | LC Bur B | REF 3711 | SD Polishing paste B |
| REF 5372 | Set-Up manual saw | REF 3712 | SD Polishing brush |
| REF 5288 | Diamond separating disc | REF 3713 | SD Felt wheel |
| REF 3370 | HM Tricutter fine (green) | REF 3714 | SD High gloss polishing mop |
| REF 3006 | Pellet cover (large) | ————— | Pink wax |

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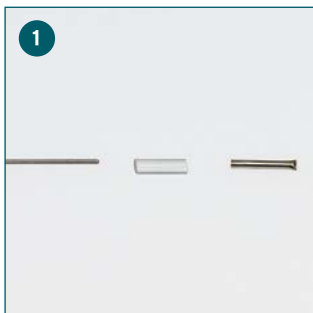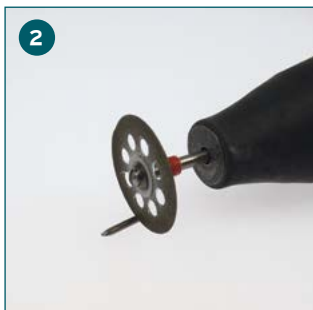Guiding rod, silicone tube,
telescope sleeve

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

The telescope sleeve has to be
adjusted individually, making sure
its length **is at least 6 mm.**

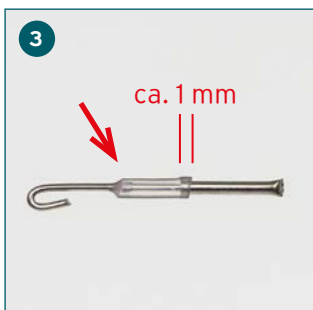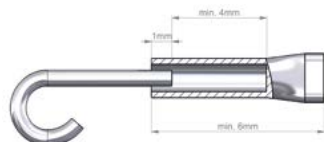The size of the silicone tube should
be about 5 mm.

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Bend a loop retention on one side
of the guiding rod. Then push the
silicone tube to the other end,
inserting it into the telescope sleeve
up to the stop.

Make sure to cover the junction
between sleeve and rod for
approx. 1 mm with the silicone tube.
Apply rose base plate wax to the
silicone tube in the area of the
junction.

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Reduce the lower part of the space
holder of the special traction screw
using the diamond separating disc.

Hint: Make sure **not** to squeeze
the space holder when reducing it
using scissors or wire cutters.

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In order to ensure easy removal of
the space holder, we recommend to
enlarge it by applying wax.

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Block out the model using e.g.
SIL-KITT red.

Then pressurize ISOFOLAN®, reduce
it at model bottom and make
occlusal incisions.

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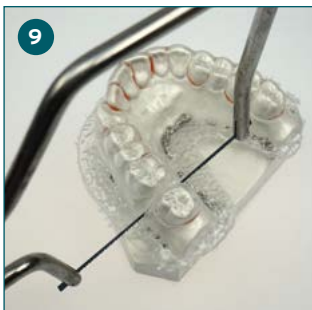
Then embed the model in the pellets and pressurize DURASOFT® pd .

Hint: We recommend using the pellet cover!

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Shorten material coarsely, not withdrawing it from the model

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Separate the pressure moulded DURASOFT® pd splint on the model centrally in the area of the interdental gap.

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Apply rose wax to the saw cut and remove wax residues.

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Use blocking out putty to protect the occlusal surface and to block out the interdental gap with blocking-out putty. Make sure to parallelly align the special traction screw and the metal parts (parallelly to each other and in relation to the occlusal plan) using a small amount of premixed BIOCRYL® RESIN and fix then (this does not require curing in a pressure pot).

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Then apply DURASPLINT® resin to the complete extensions of the construction elements and let them polymerise in the pressure pot.

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After polymerisation, withdraw the upper part of the space holder using flat pliers and remove blocking out putty.

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Expose the silicone tube on the vestibular side and remove it. Mark the guiding rod alongside the sleeve using a permanent marker.*

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***Attention!**

Make sure not to blur the black marking, as the marking will be required when proceeding with step 18 and 21.

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Make an occlusal cut/separation using manual saw or bur. Optionally, you can make the saw cut with the splint remaining on the plaster model and being removed only during the next step.

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Then carefully separate completely from below to metal parts with manual saw.

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
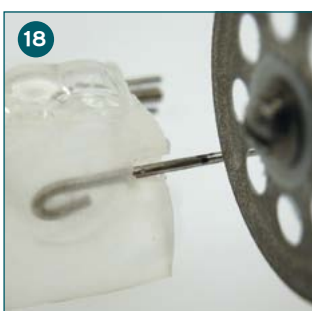
Unscrew the traction screw completely against the direction of the arrow, dismantle it and remove the rest of the silicone tube.

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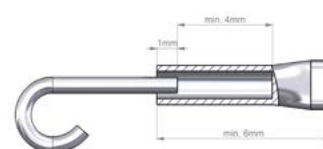Cut the guiding rod approx. 1 mm behind the marking, the rod protruding for about 1 mm max. into the telescope sleeve. This ensures there will be enough space for the guiding rod in the sleeve when tightening the screw.

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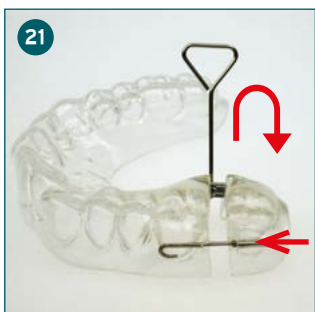
The resin in between the inter-dental gaps has to be completely removed. The future contact surfaces of the teeth have to be fully exposed.
Smooth rims and borders with the DIMO® wheels.

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The completely exposed contact surfaces will allow for optimized space closure later.

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Finally, make sure to screw together the traction screw, tighten the screw up to the black marking, ensuring a good fit of the splint on the initial model. Have the guiding rod slide simultaneously into the telescope sleeve..

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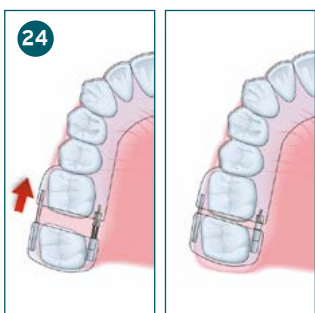
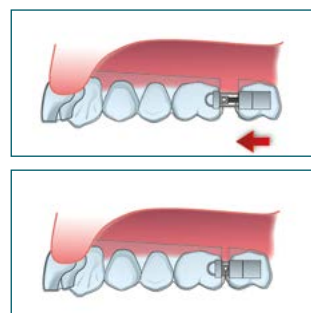
The TFA appliance can be used for space closure in the posterior region. In case of double-sided space closure, please make sure not to activate the screws simultaneously on the left and right side in order to prevent the incisors from tipping.

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Activation hint special traction screw :

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→ www.scheu-dental.com → SCHEU-ACADEMY → Themen: CA® CLEAR ALIGNER

Hint:

Please note this instruction does not replace attending our technical CA® courses.

We also recommend participating in the following additional course:

„Course No. 17 - module 5: CA® CLEAR ALIGNER splints with specific VECTOR® expansion screws“.

For further information and details on the course please visit our website:

→ www.scheu-dental.com → SCHEU-ACADEMY → Topics: CA® CLEAR ALIGNER