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Fabrication instructions for CA[®] CLEAR ALIGNER splints using VECTOR[®] 30 screws and the CA[®] CHROMIUM arch

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Fabrication instructions for CA® CLEAR ALIGNER splints using VECTOR® 30 screws and the CA® CHROMIUM arch*

Correction of malocclusion in the pressure moulding
technique becomes even more economic, effective
and time-saving in comparison to the conventional
CA® Set Up technique when using the specific
VECTOR® 30 screw and the CA® CHROMIUM arch for
the fabrication of CA® CLEAR ALIGNER splints.
Relevant literature: CA® Book Volume 2 Advanced
(Chapter: VECTOR® 30 Screw with CA® CHROMIUM
arch (English version: REF 5309, German version
REF 5307).

Working Material:

REF 2484	VECTOR® 30
REF 5317	CA® CHROMIUM Arch
REF 3207	ISOFOLAN®
REF 3407	CA® Foil hard (0.75 mm)
REF 3218	BIOCRYL®-RESIN Monomer
REF 3221	BIOCRYL®-RESIN Polymer
REF 5288	Diamond Separating Disc

*Hint: The time-saving fabrication in the pressure moulding
technique is only possible under the condition the VECTOR® 30
screws can be positioned close to the palate.

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Model Preparation:

Fix the screw position and mark separation cuts.

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Drill holes for the resin space holder using the Set Up bur.

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Make sure the screw is positioned close to the palate.

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Preparation of the CA® CHROMIUM Arch:

For easier insertion the ends of the CA® CHROMIUM arch are tapered (conical).

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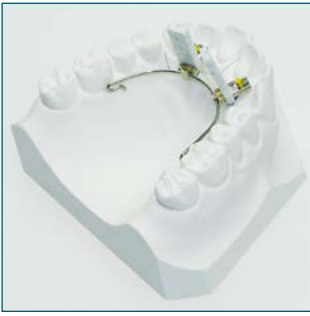
Check position and arch radius on the model.

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Reduce CA® CHROMIUM arch and bend the retention loops, making sure the retention ends can be used as selective supports on the model.

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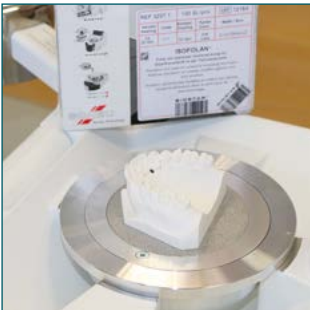
Reduce the upper part of the resin space holder by means of the diamond separating disk.

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Initial control: Positioned unit on the model.

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Pressure Moulding:

Press ISOFOLAN® foil over the model working on the model platform (the arch with screws not being on the model).

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Reduce ISOFOLAN® foil at the model rim and cut interdentally with a scalpel.

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Place the CA® Arch with the screws on the ISOFOLAN® foil using SD Cyano Veneer. Mix BIOCRYL®-RESIN until it has a creamy consistency, apply it to the construction elements and let it cure. Embed the model into the pellets, apply a small quantity of freshly mixed resin and proceed with pressurising.

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To ensure a sufficient polymerisation the material should stay under pressure in the unit for at least 15 minutes. Ignore any acoustic signals during this period, making sure not to evacuate the pressure and to switch off the machine.

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Separate the foil according to the marked extensions, working on the model.

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Then proceed with grinding the material in the area above the grey space holders with the LC Bur C.

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Extend the separation cuts to the direction of the screws/combine.

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ATTENTION:

Ensure the CA® CHROMIUM arch won't be damaged!

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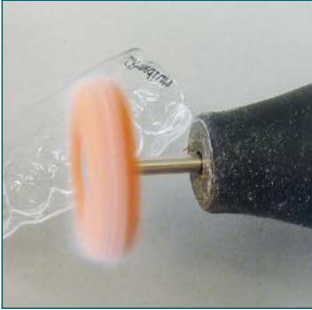
Remove the space holder with the CARAT® Flat Pliers.

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Proceed with finishing the CA® CLEAR ALIGNER splint as usual using the recommended accessories (page 1):

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Finished CA® CLEAR ALIGNER splint for oblique segment expansion according to Dr. Pablo Echarri.

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→ www.scheu-dental.com → SCHEU-ACADEMY → Themen: CA® CLEAR ALIGNER

Important:

Make sure not to exceed the maximal tooth movement in the CA[®] CLEAR ALIGNER technique acc. to Dr. P. Echarri (CA[®] Book volume 2, Advanced, pages 52/53).

Hint:

Please note this instruction does not replace attending our technical CA[®] courses.

We also recommend participating in the following additional course:

„Course No. 17 - module 5: CA[®] CLEAR ALIGNER splints with specific VECTOR[®] expansion screws“.

For further information and details on the course please visit our website:

→ www.scheu-dental.com → SCHEU-ACADEMY → Topics: CA[®] CLEAR ALIGNER

SCHEU-DENTAL GmbH
www.scheu-dental.com

phone +49 2374 9288-0
fax +49 2374 9288-90

Am Burgberg 20
58642 Iserlohn · Germany

DE/GB 200/02/17 G REF PM0160.01