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Dear Customer,

With this newsletter we'd like to inform you about the latest update for your CA<sup>®</sup>-SMART 3 software. We recommend updating of your software as soon as possible.


The following features have been changed with the actual update:

- A rare bug, which is caused by low resolutions, got fixed.
- New russian printlayouts.

Best regards,  
SCHEU-DENTAL GmbH

#### Update Instructions CA<sup>®</sup>-SMART 3.1.18:

1. Start CA<sup>®</sup>-SMART3.
2. Navigate to "System info" in the "help" menu.
3. Compare the three indicated directory paths and subdivide installation in 2 different versions:  
Version A:  
Paths are identical with the picture below and indicate express installation.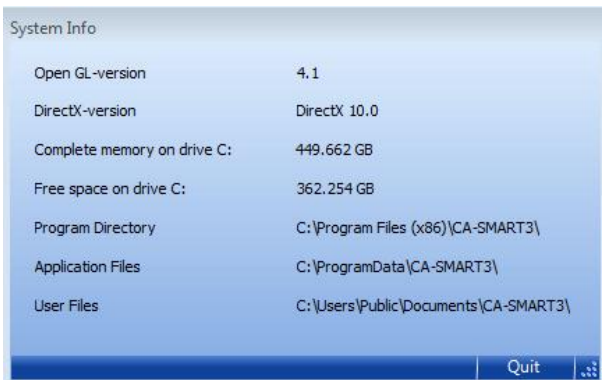

#### Version B:

Paths have been changed individually for i.e. network installations. We recommend noting all existing paths.

4. Close CA<sup>®</sup>-SMART3 and download the set-up file of the update from [here](#) and start.
5. Installation instruction for version A: Follow the instructions of the installation assistant.  
Please select express installation  
Installation instruction for version B:  
Follow the instructions of the installation assistant, but remove the checkbox after express installation.  
Enter the paths according to your notes.
6. The drivers being already installed, you can interrupt their installation by clicking "cancel". Confirm cancellation. The remaining installation is completed automatically.