CLEAR ALIGNER

Patientenhinweise / Hints for patients

Inhalt

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Contents

Introduction	24-25
The 3-Splint-System	26-27
Preparation	28-29
Inserting & removing the aligner	30-31
Eating & drinking	32-33
Cleaning & care	34-35
Wearing instructions	36-37
Storage & wearing	38-39
Attachments & IER	40-41
Warning & Remaining risks / Contraindications	42

Dear patient,

We are pleased that you received your custom-made CA[®] CLEAR ALIGNER splints from your doctor. Of course, we like to make wearing your splints as pleasant as possible for you. Therefore, we have summarized the most important notes in this small brochure. Please feel free to contact your doctor if you should have additional questions.

We wish you much success with your invisible tooth correction!


The 3-Splint-System

What makes the difference with CA[®] CLEAR ALIGNER:
Three different material thicknesses worn one after the other - for a unique and beautiful smile.
Thanks to the steady increase in pressure teeth are efficiently, yet smoothly aligned -with extraordinary comfort.
During the first week, you'll be wearing CA[®] CLEAR ALIGNER

soft before replacing it by CA[®] CLEAR ALIGNER medium in the second and CA[®] CLEAR ALIGNER hard in the third week.
Following the treatment plan of your doctor who regularly monitors the treatment progress, you'll continue then wearing the next set of splints (again soft, medium, hard).
In order to achieve optimised results, at regular intervals

new impressions are taken and new splints fabricated accordingly. Following any

further impression, you'll be wearing the CA[®] hard splint for another week.

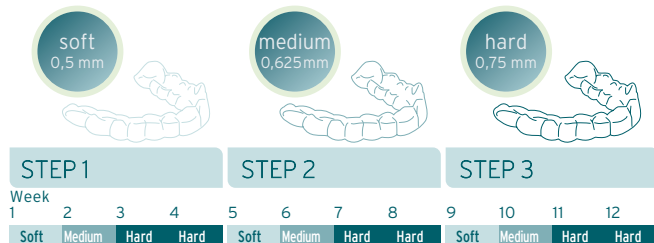
Before inserting your CA[®] CLEAR ALIGNER

You can easily remove your aligner from your mouth and insert it again, for example for eating and drinking, dental care and cleaning. Make sure to brush your teeth every time be-

fore inserting your CA[®] CLEAR ALIGNER. Before inserting the aligner for the first time, please pay attention to the following tips:

Inserting the CA[®] CLEAR ALIGNER

1. Make sure to insert the rear part of the aligner first.
2. Then proceed with pressing the front part of the splint on your teeth.

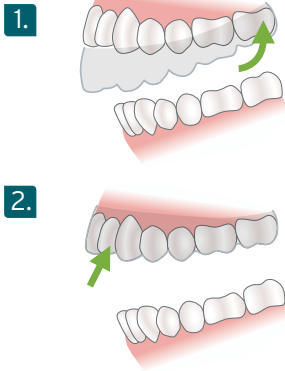
Removing the CA[®] CLEAR ALIGNER

3. Make sure to remove the rear part of the aligner first.
4. Then remove the front part of the aligner.

For easier removal we recommend using the Splint Remover.

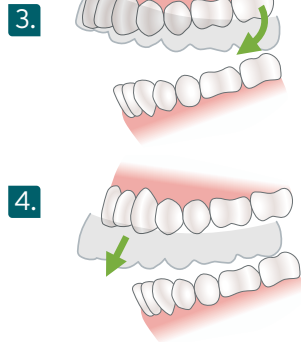You should always remove your CA[®] CLEAR ALIGNER before eating or drinking, especially when consuming hot beverages like coffee or tea.

You should also avoid chewing gum, as it might stick to the splints. Moreover, you should abstain from smoking to avoid discoloration of the splints.

We recommend cleaning the splint daily using CETRON® powder (REF 3237/3239) from our CETRON® range. The splint can also be cleaned using a denture brush and water. To prevent changes, discolouration and damage to the splint, cleaning agents containing oxidising agents (active oxygen, chlorine, etc.) should not be used.

Similarly, alcohol-based or organic solvents such as ethanol, acetone, etc. must not be used for cleaning. Cleaning the teeth or the splint with toothpaste or mouthwash containing anionic surfactants, such as sodium lauryl sulphate, should be avoided.

The material is temperature-sensitive and should not come into contact with boiling water or hot drinks such as coffee or tea.

Remember to brush your teeth every time before inserting the CA® CLEAR ALIGNER.

CA[®] CLEAR ALIGNER with expansion screw

If your doctor prescribed you a preliminary treatment with an integrated expansion screw, make sure to follow his instructions.

This kind of pretreatment for expansion with an expansion screw takes around eight weeks; the actual treatment time, however, may vary depending on the treatment goal and treatment history. Your dentist will be pleased to consult you.

VECTOR 45

VECTOR 40

Retainer box

The retainer box offers enough space for up to two CA[®] CLEAR ALIGNER splints.

You'll find the key for the incorporated expansion screw in the lid of the retainer box.

Daily Wearing Time

Your dentist will inform you in details about the daily wearing time of the CA[®] CLEAR ALIGNER splints. We recommend a minimum daily wearing time of 17 hours.

Depending on the complexity of the treatment, your dentist may have to apply small composite build-ups (attachments), so-called CA® POWER GRIP forms to your teeth, which will help to support certain tooth movements such as rotations or extrusions. Consult your dentist immediately in case an attachment gets lost during treatment.

Interproximal enamel reduction (IER)

Moreover, depending on the required space for the tooth movements your dentist may have to effect an **Interproximal Enamel Reduction (IER)**.

IER offers a gentle alternative: A very thin enamel layer is being removed using a fine diamond-coated file or oscillating devices. Subsequently, the tooth is being polished and fluoridated so it can stay healthy and stable for years.

Warning

While wearing the CA[®] CLEAR ALIGNER, avoid consuming sugary and acidic drinks. If you want to drink these kinds of beverages, remove the CA[®] CLEAR ALIGNER and make sure to brush your teeth. Consuming sugary or acidic drinks while wearing your CA[®] CLEAR ALIGNER can lead to severe tooth decay.

Remaining risks/ Contraindications

Allergic reactions to components of CA[®] Foil (PETG) cannot be ruled out in sensitised people.

Not indicated in cases of poor oral hygiene, active periodontal disease, acute intraoral inflammatory processes, or neoplastic changes. Any use not stated above is contraindicated.

SCHEU-DENTAL GmbH

Am Burgberg 20
58642 Iserlohn · Germany
phone +49 2374 9288-0
fax +49 2374 9288-90
www.scheu-dental.com

**SCHEU-DENTAL
custom-made GmbH**

Walder Straße 53
40724 Hilden · Germany
phone +49 2104 80041-0
fax +49 2104 80041-99
www.scheu-dental.com

DE/GB 1.200/04/26 G REF PM0179.01