- D Unsere Folien sind zellbiologisch geprüft und freigegeben.
- Our material has been tested and approved in terms of cell biology.
- F Nos plaques de thermoformage ont été testées et approuvées sur le plan de la biologie cellulaire.
- E Nuestras planchas están probadas y aprobadas con respecto a la biología celular.
- I nostri dischi sono testati e approvati con riguardo alla biologia cellulare.

Bedienungsanleitung **Operation Manual** Mode d'emploi Manual de Instrucciones Istruzioni per l'uso

WEEE-Reg.-Nr.: DE 52399144

Am Burgberg 20 58642 Iserlohn · Germany

phone +49 2374 9288-0

Facebook-

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# **GB** Warranty terms and conditions for the following machines of SCHEU-DENTAL GmbH

The machine you bought has been carefully checked and found to be in perfect condition, meeting highest technical demands.

SCHEU-DENTAL GmbH does not provide warranties for:

- repair services effected by the buyer or unauthorised staff
- in case of damages due to inappropriate use or handling
- in case of a failure to comply with the handling, cleaning and maintaining prescriptions
- in case of inappropriate connection or incorrect installation
- in case of force majeure or any other influences beyond the control of SCHEU-DENTAL
- in case of any modifications to the machines
- in case of using spare parts of other manufacturers
- in case of signs of wear due to age, not affecting the main functions of the machines.

Utilisation of warranties does not extend or restart the standard warranty period. In case of unjustified utilisation of our warranties we reserve to • riparazione impropria o da parte di personale non autorizzato charge a fee for the corresponding services.

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# © Conditions de garantie pour les appareils SCHEU-DENTAL GmbH
# © Condiciones de garantía para equipamientos de la empresa SCHEU-DENTAL GmbH
# Ondizioni di garanzia per apparecchiature della ditta SCHEU-DENTAL GmbH