ECHARRI PST Set

Verarbeitungshinweise

Beschreibung: Das ECHARRI PST Set besteht aus doppelt diamantierten CA® Stripping Tools in 5 Abstufungen für die

Progressive Stripping Therapie und wird in einem Alukoffer mit ie 1x CA® Stripping Tool 90 um (grün). 60 µm (grau), 40 µm (rot), 25 µm (weiß) und 15 µm (gelb) sowie 1 CA® Messlehren-Set geliefert. Das CA® Messlehren-Set (01-0.5 mm) ist aus Edelstahl (Werkstoff Nr. 1431=/AISI 301) gefertigt.

Indikation: Das Echarri PST Set ist indiziert für den Schmelzabtrag im Rahmen der Progressiven Stripping Therapie

(PST). Das CA® Messlehren-Set ist indiziert für die Überprüfung des Schmelzabtrags während der ASR.

Kontraindikation: CA® Stripping Tools mit 90 um Körnung (grün) sind **nicht** für den Schmelzabtrag indiziert, sondern nur

für die Reduktion von Füllungen und Kronen.

Unverträglichkeiten: Enthält Nickel: kann bei sensibilisierten Personen Allergien auslösen.

Dental-Produkt – nicht steril. Nur für die oben genannte Indikation und durch gualifiziertes Fachpersonal Achtung: zu verwenden. Schwerwiegende Vorfälle im Zusammenhang mit der Verwendung des Produktes sind

unverzüglich dem Hersteller und der zuständigen Behörde zu melden! Weitere Informationen zum Gebrauch des ECHARRI PST Sets finden Sie unter: www.scheu-dental.com/echarri-pst-set

und unter dem QR-Code unten.

Hinweis: Die CA® Stripping Tools sind möglichst mit EVA-Handstück zu verwenden. Bei engen Kontaktpunkten

ist vorab die Verwendung der CA® Strip Opener (REF: 5326 & 5327) zur Eröffnung eines interdentalen

Spaltes indiziert.

Die CA® Messlehren sind im Autoklaven bis max. 135 Grad Celsius sterilisierbar. Eine vorgeschaltete

mechanische Reinigung im Ultraschallbad kann ggf. erforderlich sein.

ECHARRI PST Set

Instructions for use

Description: The ECHARRI PST set consists of double diamond-coated CA® Stripping Tools in 5 gradations for pro-

gressive stripping therapy and comes in an aluminium case with 1x CA® Stripping Tool 90 µm (green).

60 μm (grey), 40 μm (red), 25 μm (white) and 15 μm (yellow) as well as 1 CA® gauge set. The CA® gauge set (01-0.5 mm) is made of stainless steel (material no. 1431=/AISI 301).

Indication: The ECHARRI PST set is indicated for enamel removal as part of Progressive Stripping Therapy (PST).

The CA® gauge set is indicated for checking the enamel removal during the ASR.

Contraindication: CA® Stripping Tools with 90 µm particle size (green) are not indicated for enamel removal, but only for

the reduction of fillings and crowns.

Intolerances: Contains nickel; may cause allergies in sensitised persons.

Attention: Dental product - non-sterile. Only for the above indication and for use by qualified professionals.

Serious incidents in connection with the use of the product must be reported to the manufacturer and

the competent authority immediately!

Further information on the use of the ECHARRI PST Set can be found at:

www.scheu-group.com/en/products/echarri-pst-set

and under the QR-Code below.

Hints: The CA® Stripping Tools should be used with an EVA handpiece if possible. For narrow contact points,

the use of the CA® Strip Opener (REF: 5326 & 5327) is indicated ton open an interdental gap.

The CA® gauges can be sterilised in an autoclave up to max. 135 degrees Celsius.

A preceding mechanical cleaning in an ultrasonic bath may be necessary.



Manufactured by: OSUNG MND CO.,LTD. 57, Hwanggeum-ro 109 beon-gil, Yangchon-eup, Gimpo-si, Gyeonggi-do, Korea



EU-Representative: LITDENT UAB A. Mickeviciaus g. 2a LT-76341 Siauliai, Lithuania

CA® CLINICAL PAPER 21:





Distributed by: SCHEU-DENTAL GmbH www.scheu-dental.com

Am Burgberg 20 58642 Iserlohn · Germany phone +49 2374 9288-0 fax +49 2374 9288-90



