

VECTOR® 900

Gebrauchshinweise/Hints for use

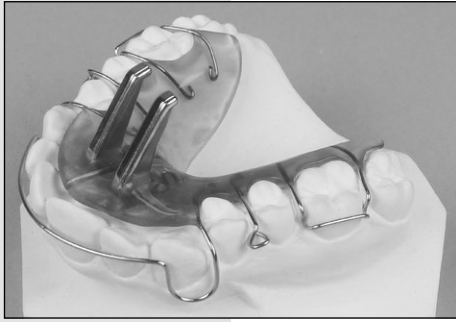
Lagerichtiger Einbau in der OK-Dehnplatte
Correct insertion in the upper expansion plate

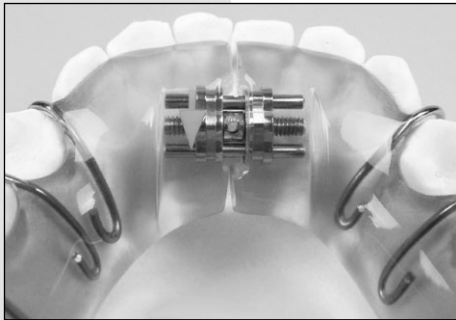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Lower expansion plate with inserted VECTOR® 100 and integrated guiding surface for the guiding bars of VECTOR® 900

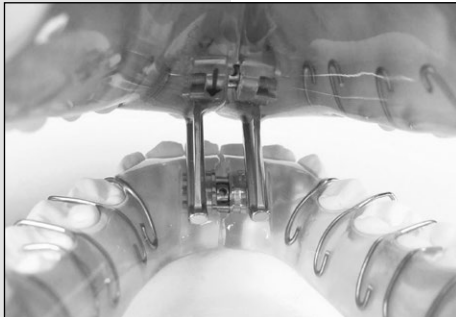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Correct insertion (70°-60° relating to the occlusal plane) of the guiding bars using the angle tester acc. to Adamik

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Special screw for insertion in protrusive double plates for treatment of Angle class II cases (lower jaw retrusion/distal occlusion) as well as for bilateral expansion of the upper jaw in growing children.

Compact design with integrated expansion screw, incl. security expansion screw and SD- box. For lower jaw we recommend the combined use with VECTOR® 100 (REF 2492). For fixing and controlling the insertion the angle tester (REF 2332) is best suited.

W x H x L: 14,0 x 20,0 x 13,0 mm

Max. expansion:

5.0 mm ~ 0.0875 mm / ¼ U. ~ 57 activations / 4x1/4 rot. = 0.35 mm

Contraindications

The used stainless steel alloy is highly corrosion-resistant and suited for use in the mouth. In patients, hyper-sensitive to nickel, however, the minor nickel percentage may cause allergic reactions. If the allergic reaction is confirmed by the corresponding allergy test, it may be necessary or discontinue treatment.

Hints for patients

We recommend our CETRON® care products for cleansing the appliances.

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