


BIOPLAST® mouthguard

for increased safety in recreational and professional sports




For any challenge:

Three mouthguard types

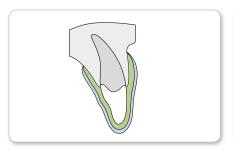


BIOPLAST® MASTER:

- for all recreational sports
- proven, two-layer mouth guard for high protection and comfort
- many colours and patterns available

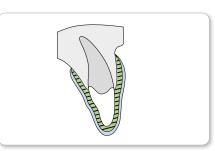

BIOPLAST® XTREME/DECO:

- suited for contact sports, as it protects against hard and concentrated hits
- with integrated reinforcement section in the front teeth area
- double-layer laminate construction for best protection and comfort

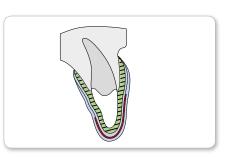

BIOPLAST® XTREME PRO:

- for professional contact sports with expected concentrated hard hits (impact is distributed across a greater area)
- four-layer construction for the best protection and comfort
- with integrated reinforcement area and supplemental hard-elastic insert around the front teeth area

BIOPLAST® MASTER

There are no limits to individualization: a variety of colours and patterns is available. Logos and names can also be inserted if required, avoiding teammates mixing up their mouth guards.



-0

** Base material. For adequate and sufficient protection, a second layer has to be laminated on top of the first one.

BIOPLAST® COLOR**

// 3.0 x 125 mm*	O clear	3187
// 3.0 x 125 mm*	red	3382
// 3.0 x 125 mm*	O blue-transparent	3356
// 3.0 x 125 mm*	ored-transparent	3355
// 3.0 x 125 mm*	O white	3445
// 3.0 x 125 mm*	black	3448
// 3.0 x 125 mm*	neon red	3352
// 3.0 x 125 mm*	neon green	3353
// 3.0 x 125 mm*	azure blue	3354
// 3.0 x 125 mm* Color-Set	One sheet each of clear, blue-transparent, red-transparent, yellow, red, white, black, neon red, neon green, azure blue.	3271

* 10 pcs.


Material in yellow is included in the Color Set; it is not available separately.

BIOPLAST® MULTICOLOR**

// 3.0 x 125 mm*	Germany	3390
// 3.0 x 125 mm*	• France	3391
// 3.0 x 125 mm*	● Italy	3392
// 3.0 x 125 mm*	Spain	3393
// 3.0 x 125 mm*	Sweden	3394
// 3.0 x 125 mm*	■ Brazil	3395
// 3.0 x 125 mm*	◆ Transylvania	3396
// 3.0 x 125 mm*	Atlantis	3397
// 3.0 x 125 mm*	White Dots	3398
// 3.0 x 125 mm*	Green Dots	3399
// 3.0 x 125 mm*	O Multicolor Set	3400

BIOPLAST® XTREME/DECO

One pressure moulding sheet – two different degrees of hardness!

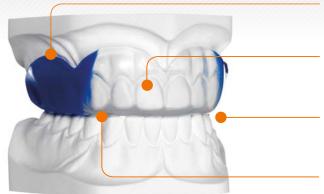





In addition to the two-color material, a new manufacturing technology also enables photorealistic, multi-color designs.

BIOPLAST® XTREME PRO

This four-layer construction comes with an additional hard-elastic layer for integrated reinforcement in the front tooth area.



Lateral segment, coloured, of BIOPLAST® 3.0 mm Shore 80 A

Frontal reinforced segment, clear-transparent, of BIOPLAST® XTREME 3.0 mm Shore 92 A

Thanks to the **V-shape** of the plate, by changing the position of the model you can vary the size of the reinforced anterior section for individual patients.

Laminated cover plate, clear-transparent, of BIOPLAST® 2.0 mm Shore 80 A resp. 1.0 mm (BIOPLAST® XTREME PRO)

BIOPLAST® XTREME

// 4.0 x 125 mm*	red	3295
// 4.0 x 125 mm*	blue	3296
// 4.0 x 125 mm*	black	3297
// 4.0 x 125 mm*	○ clear	3298
// 5.0 x 125 mm*	red	3286
// 5.0 x 125 mm* // 5.0 x 125 mm*	red blue	3286 3287
	- 134	0200
// 5.0 x 125 mm*	blue	3287

* 10 pcs. only for MINISTAR® beginning with 4/92 and BIOSTAR® 4/99

1/* BIOPLAST® XTREME Kit* 1x BIOPLAST® XTREME (5.0 x 125 mm), 1 bag of 15 g CETRON® cleansing powder, 1 BIOPLAST® insulating agent, retainer box	• red	3283
✓ BIOPLAST® XTREME Kit* Content as above	blue	3284
// BIOPLAST® XTREME Kit* Content as above	black	3285

* only for MINISTAR® beginning with 4/92 and BIOSTAR® 4/99

BIOPLAST® XTREME DECO

// 5.0 x 125 mm*	chinese	3312
// 5.0 x 125 mm*	dragon	3313
// 5.0 x 125 mm*	kiss	3314
// 5.0 x 125 mm*	aqua	3315
// 5.0 x 125 mm*	stone	3316
// 5.0 x 125 mm*	lion	3317

 * 5 pcs. only for MINISTAR® beginning with 4/92 and BIOSTAR® 4/99

BIOPLAST® XTREME PRO

**DIOPLAST® XTREME PRO Kit* 1x BIOPLAST® EXTREME (4.0 x 125 mm), 1x BIOPLAST® clear (2.0 x 125 mm), 1x DURAN® (0.75 x 125 mm), 1 bag of 15 g CETRON® cleansing powder, 1x BIOPLAST® insulating agent, retainer box	• red	3292
// BIOPLAST® XTREME PRO Kit* Content as above	blue	3293
// BIOPLAST® XTREME PRO Kit* Content as above	black	3294

Retainer Boxes

For clean and hygienic storing of BIOPLAST® mouth guard.

Available in five colours.

// Retainer Boxes

5435

* 10 pcs.



Finishing Set

Assortment containing 8 grinding instruments with colored marking for easy identification – for cutting and finishing soft elastic and hard elastic material.

// Finishing Set

Set comes with cutting bur, finishing bur, HM Carbide cutter, BIOPERM Trimmer, POLYFIX, OSAMU polisher, DIMO®PRO SLIM, pulp stone

3378



Trimmer and Polisher

For optimal trimming and polishing of BIOPLAST® material. BIOPERM-Trimmer for finishing the edges. POLYFIX for exposure of frenula. OSAMU-Polisher for final polishing.

// BIOPERM-Trimmer	blue, 2 pcs.	3226
// POLYFIX on mandrel	brown, 1 pc.	3371
// POLYFIX	brown, 6 pcs.	3372
// OSAMU-Polisher	grey, 2 pcs.	3247



DIMO®PRO

Trimming wheel made of a latex filled abrasive with long working time for BIOPLAST® material, incl. mandrel

// DIMO®PRO	wheel 24 x 6 mm	3381
// DIMO®PRO SLIM*	wheel 24 x 4 mm	3376
10/50 pcs. * Tapered edge.		



BIOPLAST®-Insulating agent

Insulating agent for plaster models when working with BIOPLAST® material, especially when fabricating positioners and mouth guards. For absolute transparency and smooth inner surfaces.

// BIOPLAST-Insulating agent, 500 ml

3189



Print media

Information material for patients, available in English and German.

// BIOPLAST® Flyer	0308
// BIOPLAST® Poster	0388

Details in regard to the fabrication of a BIOPLAST® mouth guard can be taken from our application booklet or fabrication video at www.scheu-dental.com.



