

Cleansing and Care

for orthodontic appliances, splints, dentures and sports mouthguard



Mix the contents of one sachet with 150 ml water.



Leave to act for approx. 30 min. and rinse with clear water.



Any remaining sediment can be removed with toothpaste and toothbrush.



The care spray refreshes the appliance before insertion.

Due to contact with food, drinks, pharmaceuticals and individual mouth environment, splints, orthodontic appliances, dentures and sports mouthguard may lead to deposits and changes in colour, taste and smell. With CETRON® Cleansing Powder, you, or your patients, get an effective cleaning agent, while CETRON® Care Spray, with its pleasant lemon aroma, ensures long-lasting freshness and cares for your splints, braces and prostheses. The inexpensive cleansing powder ensures long wearing comfort and high patient compliance. You can easily clean and remove sediments from any kind of denture and orthodontic appliance as well as sports mouthguard, with the exception of prosthetic works made of polyamide, e.g. Valplast or Brelflex.

Delivery Program

// CETRON® Care Set	1 box with 5 bags (15 g)	3236.1
// Refill package	5 bags (15 g)	3239.1
// Refill package	25 bags (15 g)	3239.2
// Refill package	250 bags (15 g)	3239.3
// CETRON® Care spray	75 ml	3238.1

