BIOPLAST® bleach

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BIOPLAST® bleach

Processing Information

Description:	BIOPLAST® bleach is a soft elastic rubber-like thermoforming material made of ethylene vinyl acetate (EVA) that does not bond to acrylic. The material does not require predrying before pressure moulding, as it is not hygroscopic. The material has been tested and approved for its biological safety.
Indication:	Due to its material characteristics, BIOPLAST® bleach is ideally suited for bleaching splints and medical trays. The material has a higher degree of stability than the standard BIOPLAST® material, ensuring a more precise marginal adaptation.
Contraindication:	Allergic reactions to components of BIOPLAST® bleach (EVA) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.
Preparing the model:	Apply BLUE-BLOKKER® that serves as a space holder in the required layer thickness to the surfaces that you want to bleach or treat. We recommend using BIOPLASTIC® insulating agent (REF 3189/3166) for model isolation. Apply one thin layer of the insulating agent to the model, making sure to prevent the liquid from accumulating in the cavities as well as from drying out.
Pressure moulding:	Generally speaking, we recommend to place dental arches on the model and to embed full models into the pellets. Adhesion of pellets to the model and the resulting loss of pellets can be considerably reduced by using the pellet cover (REF 3006 large/REF 3007 small).
Finishing:	You can reduce the splint to its final extension using the foil scissors A (REF 3460) and DIMO® PRO SLIM (REF 3376).
Cleansing/care:	Our CETRON® cleansing and care products are the best choice for splints made of BIOPLAST® bleach.
Note:	The product must only be used for the indication specified above and by qualified personnel. For single use. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

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