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Processing Information

Description:

Space maintainer for quick and easy production of air slots in a positioner. Depending on the required space, the space maintainers can be superposed. The central recess of the space maintainer is used to support / stabilize the melted BIOPLAST® material.

Indication:

Positioner

Application:

Two-layer Positioner:

The occlusal surfaces of the BIOPLAST® material are heated on the upper or lower model using the infra-red heater and the heating support. Once heating is completed, the models are being reset into the articulator. The space maintainer is placed in the front as required and the material is laminated. The handle can be shortened if necessary (incisal pin). During cooling, the vertical height of the articulator should be secured with a rubber band. The space maintainer can be pulled out after cooling. The marginal areas are finished using the BIOPERM trimmer or the DIMO®PRO wheel before polishing them with the OSAMU polisher.

Three-layer Positioner:

In this version, after laminating the material the space maintainer is not yet being removed, but reduced after cooling. Then the laminated material blanks are removed from the model, reduced and some perforations are made from the vestibular side as well as interdentially. Subsequently, the blanks are set back on the models and cleaned. The models are embedded in the pellets in the BIOSTAR® device in an upright position.

Thermoforming and Finishing:

The third material blank is now thermoformed in the pressure moulding technique. Once the material is cooled, the space maintainer can be exposed using the BIOPERM trimmer or DIMO®PRO wheel before removing it. The positioner is finalized according to the description in the application booklet for the pressure moulding technique.

Delivery program:

Space maintainer for positioner (trapezoidal)	2 pcs.
Description	
Space maintainer for positioner (trapezoidal)	REF W0033.001

Further products:

Space maintainer for positioner	2 pcs.
Description	
Space maintainer for positioner (semicircular)	REF W0034.001