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EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of aligner splints (for correcting tooth and jaw mis-alignments) and retention splints. Instructions for use can be found at www.scheu-dental.com/en/download.

Contraindication: Allergic reactions to compo-nents of UNIQ/UNIQ+ (copolyester) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Note: Must only be used by qualified personnel and within 15 minutes after opening. Only open the barrier bag if it is undamaged and still sealed. For single use. Not suitable for constructing splints with acrylic resin. Any serious incident involving the product must be reported to the manufacturer and the competent authority imme-diatly!

Safety information: The practitioner is responsi-ble for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using UNIQ with 3D-printed models, a non-perfor-ated protective foil (ISOFOLAN®) or alternatively UNIQ+ must be used. Remove the protective foil after the thermofforming process. Do not bring the products into contact with alcoholic/organic solvents.

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