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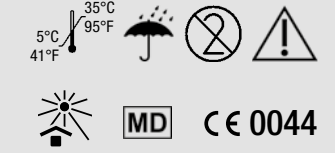
 Facebook
[scheu.dental](https://www.facebook.com/scheu.dental)

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of dental splints in different areas of application such as aligner splints, retention splints, bite splints and mandibular protrusion splints. Indication table with further indications and instructions for use at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of DURAN®/DURAN®+ (PETG) cannot be ruled out in sensitised people.

Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. For single use. Not suitable for partial build-up with light-curing resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using DURAN® with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) or alternatively DURAN®+ must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets.

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