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GA 2.000/01/25 G REF PN5398.01

## EN Instructions for use

**Intended purpose:** Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatment

**Indication:** Suitable only for the fabrication of aligner splints (for correcting tooth and jaw mis-alignments) and expansion aligners.

Instructions for use can be found at [www.scheu-dental.com/pressure-moulding-material](http://www.scheu-dental.com/pressure-moulding-material).

**Contraindication:** Allergic reactions to components of CA Foil (PETG) cannot be ruled out in sensitised people.

Any use not stated above is contraindicated.

**Storage:** Store at 5°C to 35°C. Protect from sunlight. Keep dry.

**Important:** Must only be used by qualified personnel. For single use. Not suitable for partial build-up with light-curing resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

**Safety information:** The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using CA Foil with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets.

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