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Facebook
scheu.dental

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatment.

Indication: Suitable only for the fabrication of discolouration-resistant inner layers for DURASOFT® pd splints. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of DURASOFT® seal (TPU) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. For single use. Not suitable as a single foil for the fabrication of ready-to-use splints. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/ contains bubbles; there is a risk of swallowing or aspiration. When using DURASOFT® seal with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) must be used. Remove the protective foil after the thermforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets.

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