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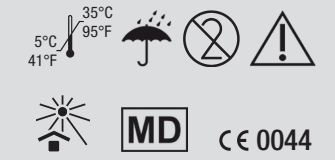
 Facebook
[scheu.dental](https://www.facebook.com/scheu.dental)

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatment.

Indication: Suitable only for the fabrication of mandibular protrusion splints, occlusal splints, bruxism splints, Miniplast splints, retainers, retention splints, long-term retainers and orthodontic correction splints. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of DURASOFT® pd (PETG and/or TPU) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. For single use. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. Only open the barrier bag if it is undamaged and still sealed. Use within 15 minutes after opening. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using DURASOFT® with 3D-printed models, a non-perforated protective foil (ISOFLAN®) must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets. The maximum temperature of 175°C must not be exceeded when using thermoforming devices from other manufacturers.

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