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 Facebook
[scheu.dental](https://www.facebook.com/scheu.dental)

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of aligner splints (for correcting tooth and jaw misalignments).

Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of CA[®] Pro/CA[®] Pro+ (copolyester and/or TPU) cannot be ruled out in sensitised people.

Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. Not suitable for build-up with resins. For single use. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

Safety information: The practitioner is responsible for correct use. Only open the barrier bag if it is undamaged and still sealed. Process immediately after opening at an ambient temperature of max. 21°C and max. 45% relative humidity. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using CA[®] Pro with 3D-printed models, a non-perforated protective foil (ISOFLAN[®]) or alternatively CA[®] Pro+ must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets. The maximum temperature of 175°C must not be exceeded when using thermoforming devices from other manufacturers.

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