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
 Facebook
scheu.dental

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of aesthetic splints. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of DURAN®+ white pd (PETG) can not be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel and within 15 minutes after opening. Only open the barrier bag if it is undamaged and still sealed. For single use. Not suitable for partial build-up with light-curing resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets.

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