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 Facebook
[scheu.dental](https://www.facebook.com/scheu.dental)

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of OSAMU-Retainers®, positioners, sports mouthgards and duplicating forms for situation models.Instructions for use can be found at www.scheu-dental.com/ pressure-moulding-material.

Contraindication: Allergic reactions to components of BIOPLAST® (EVA) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 30°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. For single use. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/ contains bubbles; there is a risk of swallowing or aspiration. Only suitable for use with plaster models. Do not bring the products into contact with alcohol-based/organic solvents, hot water (> 50 °C) or steam jets.

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