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Facebook
scheu.dental

EN Instructions for use

Intended purpose: Components for the manufacture of orthodontic appliances for palatal expansion.

Indication: The screws are used for transversal expansion of the jaws in fixed orthodontics. Instructions for use can be found at www.scheu-dental.com/en/vector.

Patient target groups: Patients who require orthodontic treatment for a narrow jaw. The orthodontic treatment is the responsibility of the treating physician and concerns both the type of therapy and the patient age at which the physician plans and carries out the treatment/therapy.

Contraindication: Allergic reactions to components of the product (nickel) cannot be ruled out in sensitised people. Not suitable for surgically assisted rapid palatal expansion. Any use not stated above is contraindicated.

Storage: No special storage conditions.

Important: Must only be used by trained personnel in the dental field. For single use. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. Protect screw body from excessive heat to avoid damaging the integrated polyamide bushing and jamming of the screw.

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