




Repair Manual for BIOSTAR® 4-Aligner

1 Replacement fuse 15A 3030.935

Replacement fuse 15A 3030.935


- 1** DISCONNECT THE DEVICE FROM THE MAINS!

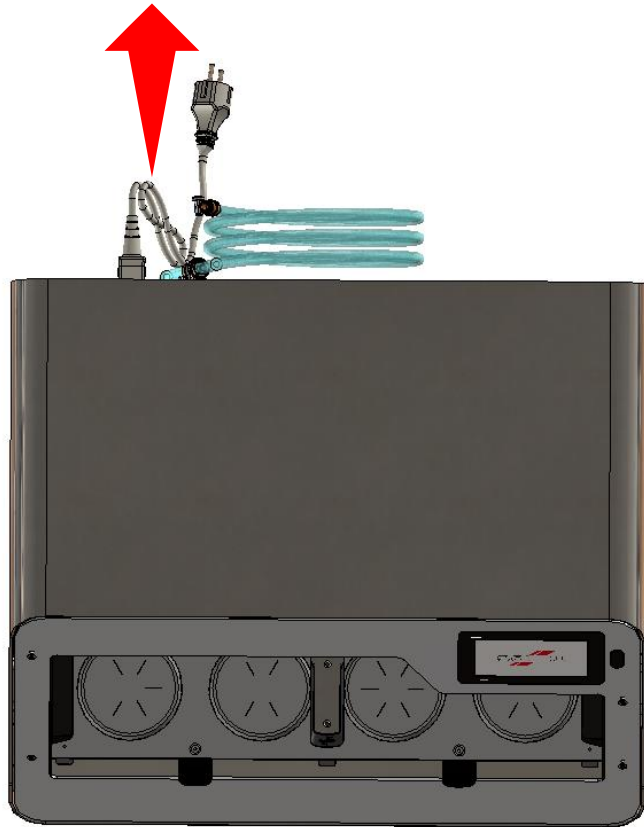

- 2** Turn the device over so that the back of the device is freely accessible.

Do NOT lift the device by the FRONT PANEL but reach under the device!

Make sure that the current software version is installed.

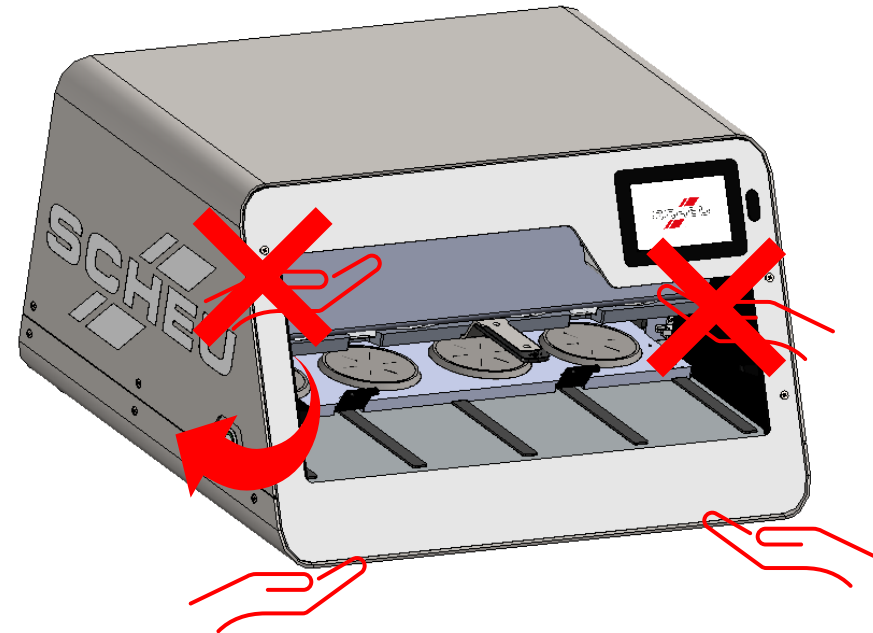

Replacement fuse 15A 3030.935

- 3** Turn the device carefully so that you can reach the back of the device plug.
- 4** Lever out the insert in the centre of the plug and replace the defective fuse.

