Repair Manual for BIOSTAR® 4-Aligner

4 Repair instructions replacement
heater / thermocouple

Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them

1

Before taking any further action, please first perform a heater test via the maintenance menu.
This is to determine whether the heater is still functional.


- Go to the Settings menu 
- Select the 'Maintenance' menu item
- Perform the heater test, follow the instructions on the display and restart the device after a successful test.

Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them


2 DISCONNECT THE DEVICE FROM THE POWER SUPPLY!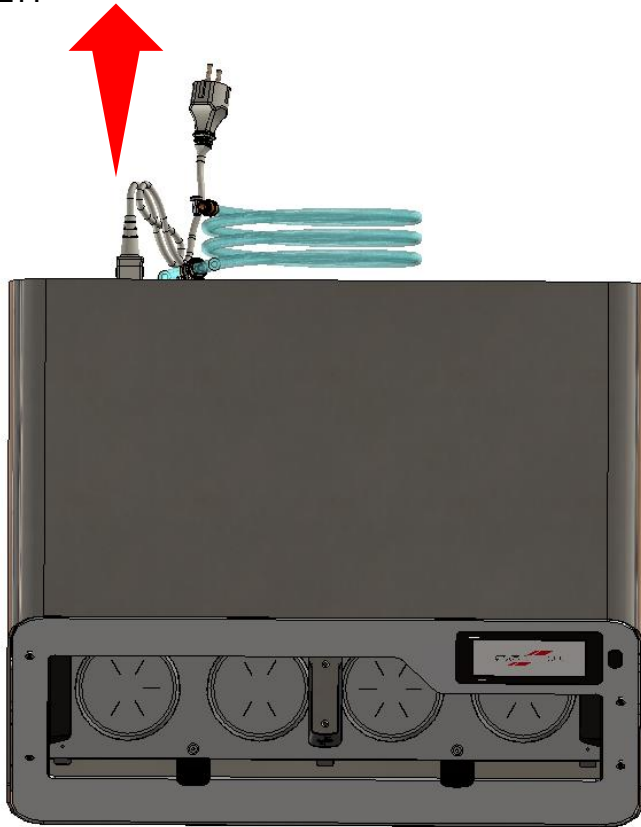

3 Turn the appliance so that you can reach the screws on both sides.

Do NOT lift the device by the FRONT panel but reach under the device!



Make sure that the current software version is installed.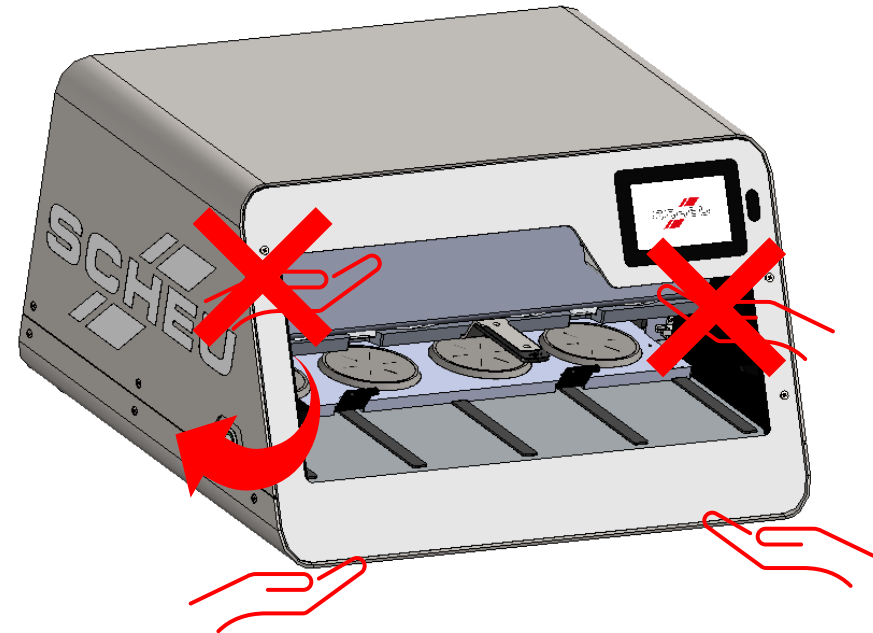

Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them

- 4** Loosen the 6 screws on the back panel and put the back panel to one side.
- 5** Pull the base plate with the heater back as far as it will go.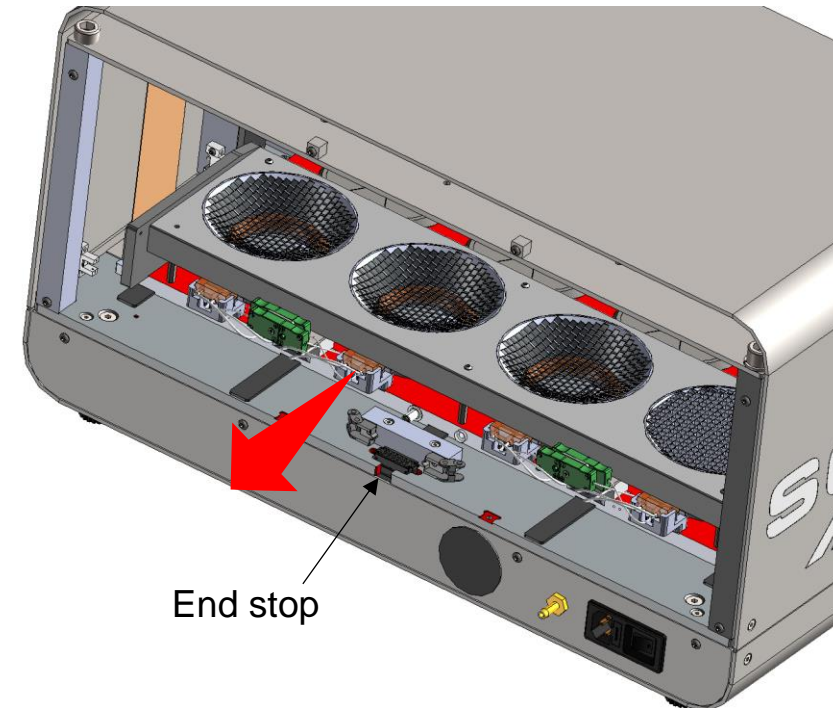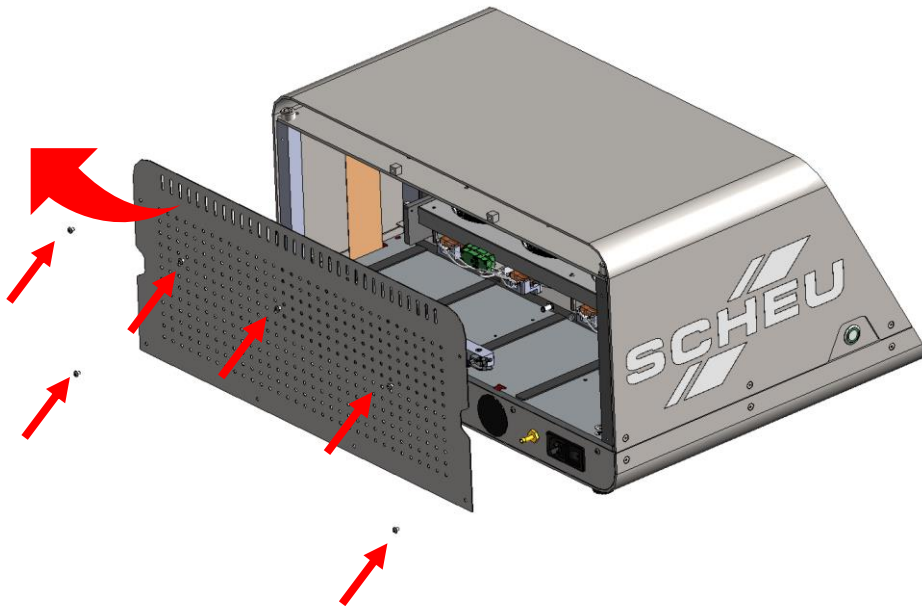


Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them

6


Loosen the 6 screws on the heater protection screen and put it to one side.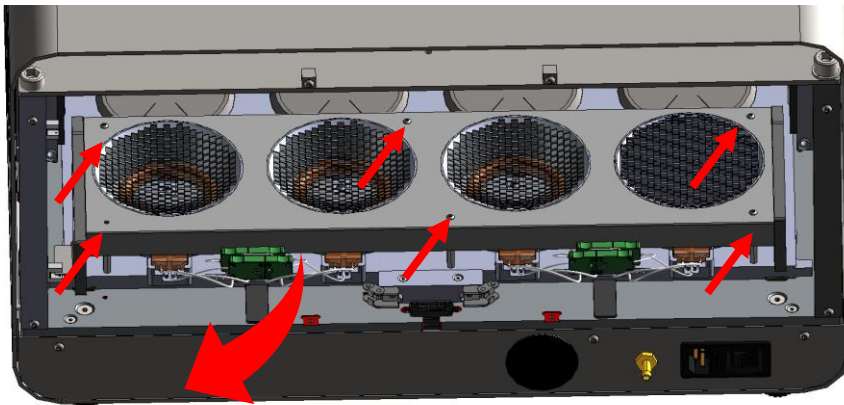

7

Select the heater you want to change.

- Loosen the clamping screws in the porcelain clamp that contacts the cable and pull out both cables of the heater.
- Slightly stretch the spring holding the heater and carefully lift the heater out.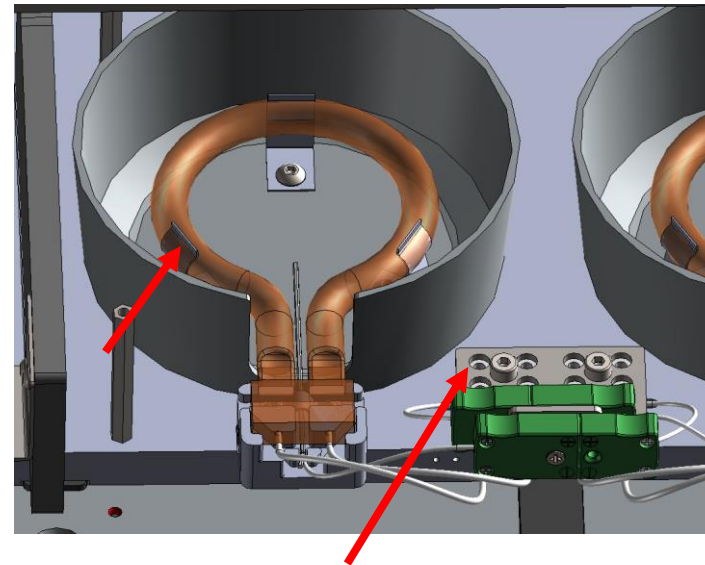


Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them

8

If you want to replace the thermocouple, the heater must be removed as explained up to this point. Loosen the two screws of the fastening clip and remove it.



Loosen the screw of the plug for the thermocouple (green), pull it out and insert the new thermocouple. Position the thermocouple so that it is centred to the reflector.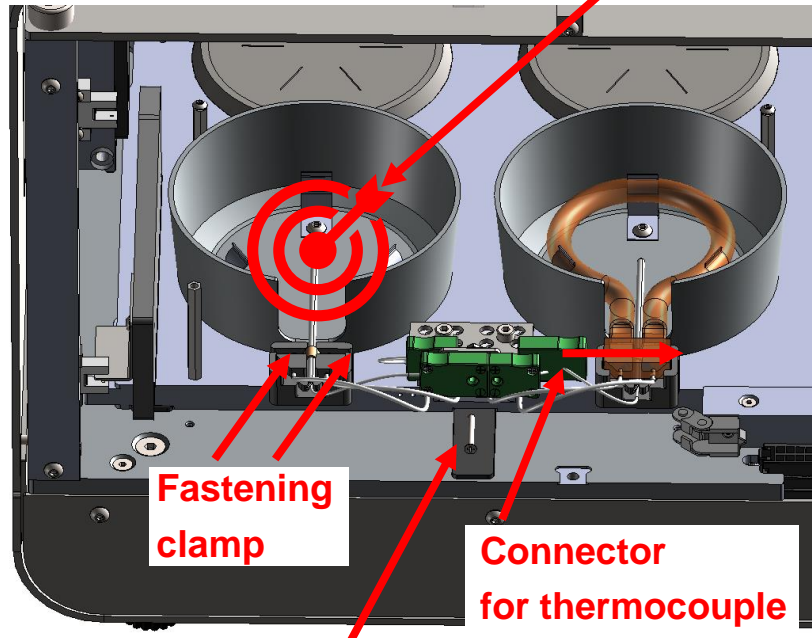

9

Reinstall the new heater or the one removed to replace the thermocouple, preferably wearing gloves. To do this, bend the springs apart and carefully clamp the heater.

Ensure that the heater is installed with the side of the short cable facing the porcelain clamp.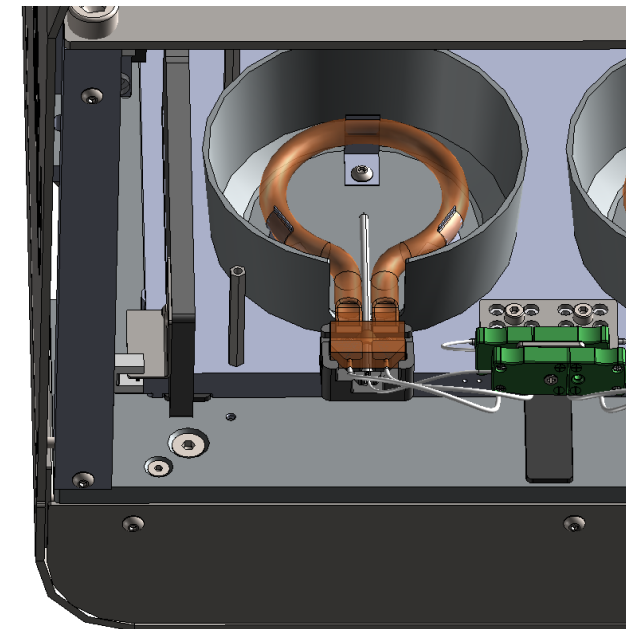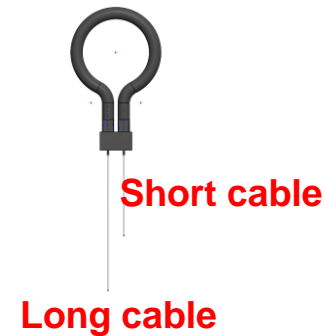


Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them



10

Finally, fit the protective screen 3030.700 for the heater and screw on the rear panel. Make sure that the long side of the protective screen is facing the back wall when inserting it.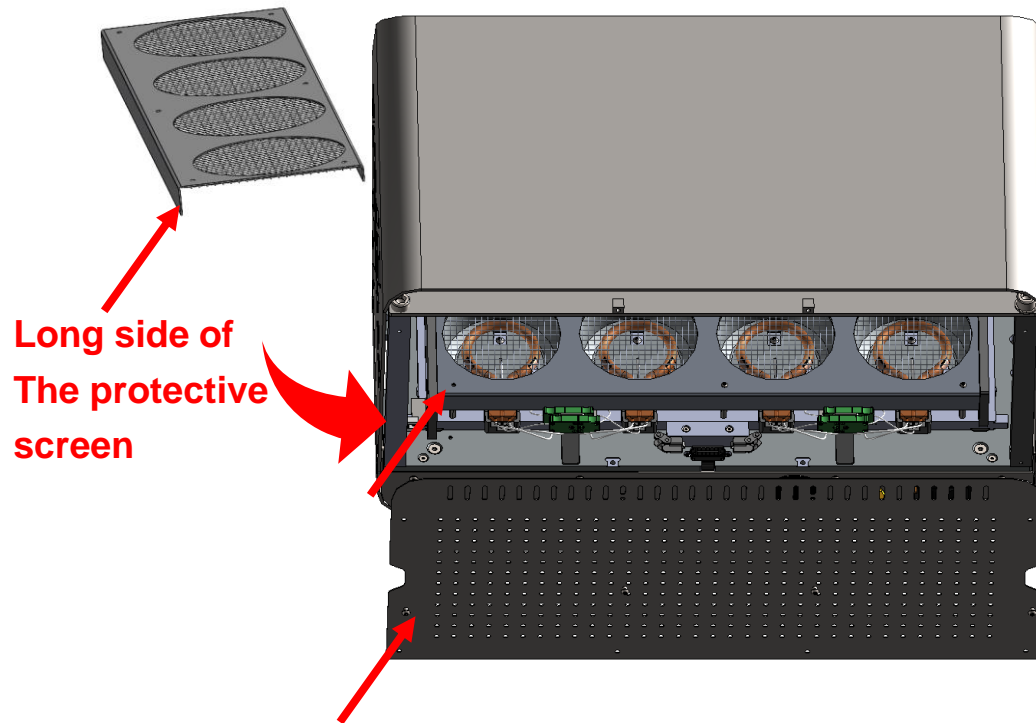

11

Perform the heater test in the software programme to check whether everything has been installed correctly.

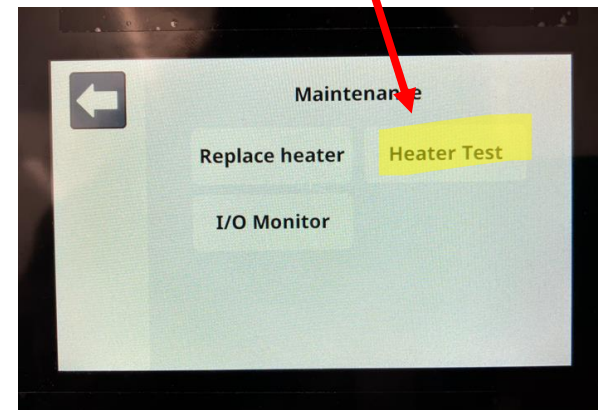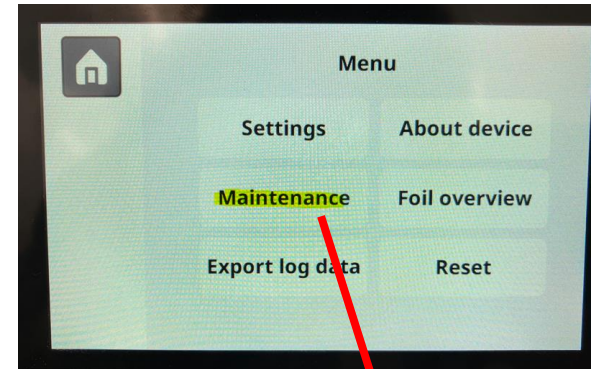Repair instructions replacement heater / thermocouple

ERROR 002 Heater 3030.916R or thermocouple 3030.827 defective → Replace them

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