




Repair manual BIOSTAR® 4-Aligner

6 CMOS battery replacement

CMOS battery replacement

ERROR 007 Die The battery voltage is below 2,8 V → Replace the CR2032 button cell


1 DISCONNECT THE DEVICE FROM THE POWER SUPPLY!

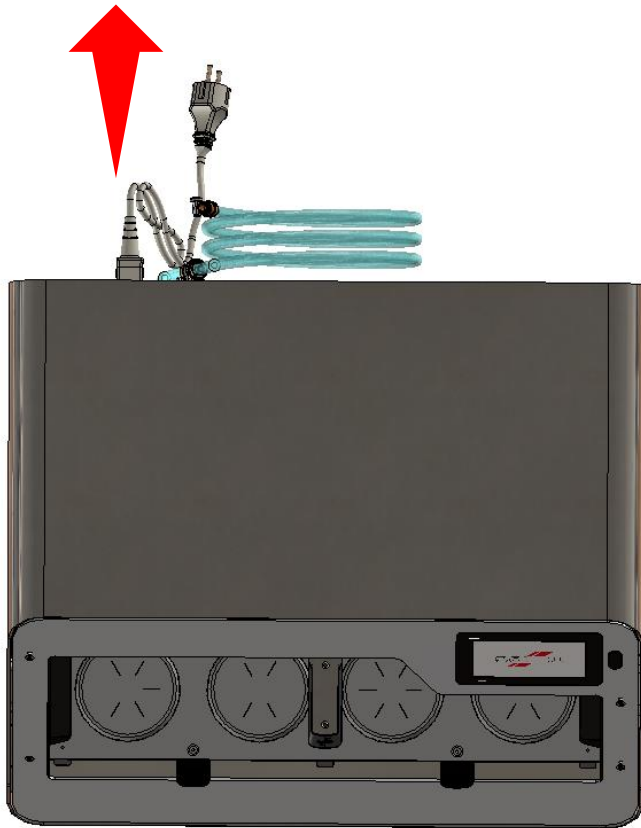

2 Turn the appliance so that you can reach the screws on both sides.

Do NOT lift the device by the FRONT panel but reach under the device!


Make sure that the current software version is installed.

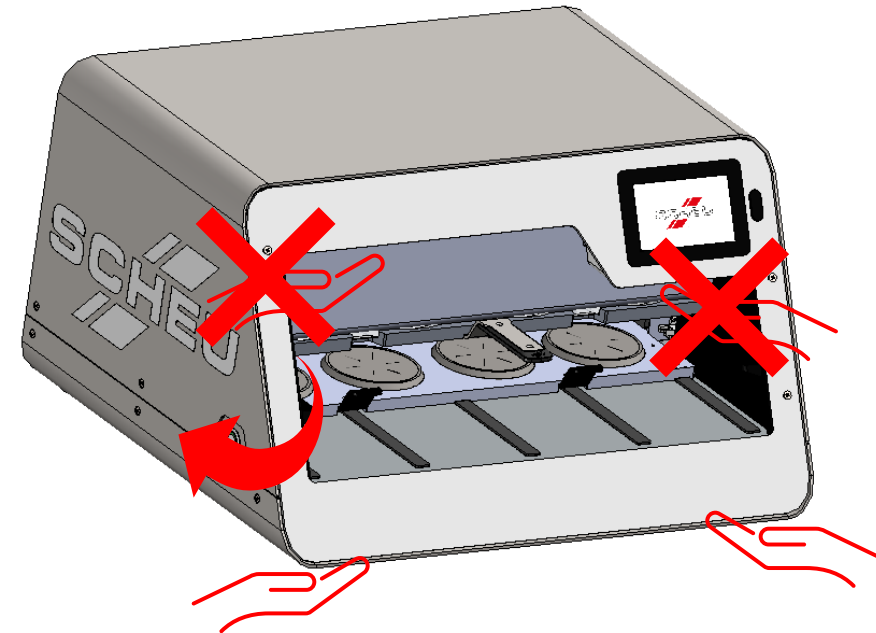


CMOS battery replacement

ERROR 007 Die The battery voltage is below 2,8 V → Replace the CR2032 button cell

- 3** Remove the 6 screws from the rear panel. Then press on the side of the upper part of the housing to carefully detach the ribbon cable from the double-sided adhesive tape on the upper part of the housing.

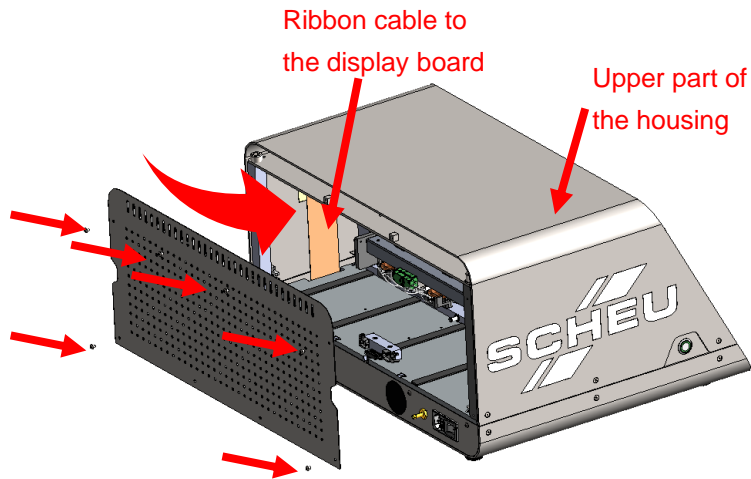

- 4** Remove the 6 screws from the upper part of the housing. Caution! If necessary, remove the ribbon cable from the adhesive tape again.

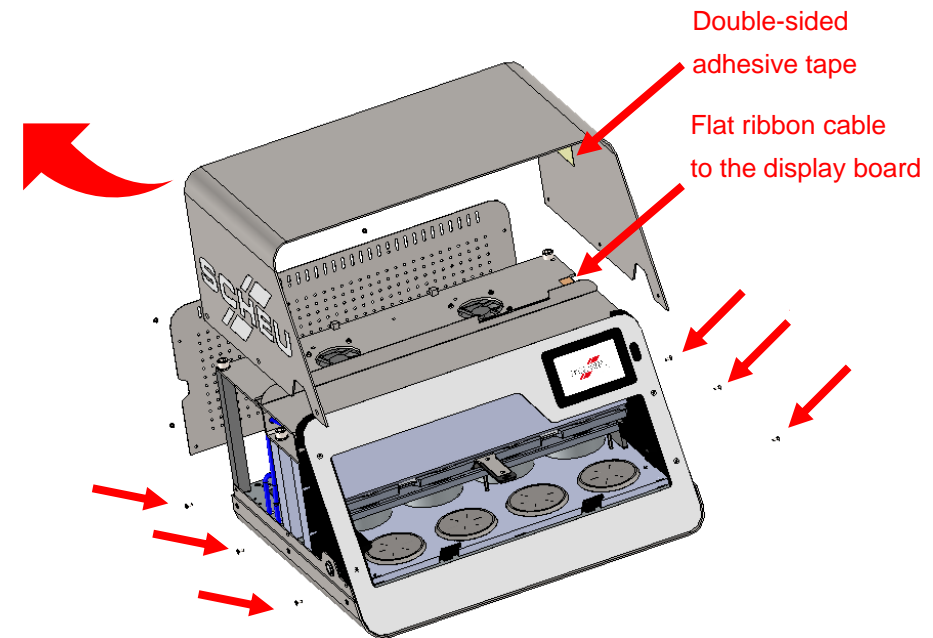


CMOS battery replacement

ERROR 007 Die The battery voltage is below 2,8 V → Replace the CR2032 button cell

5

The battery is located here for replacement.

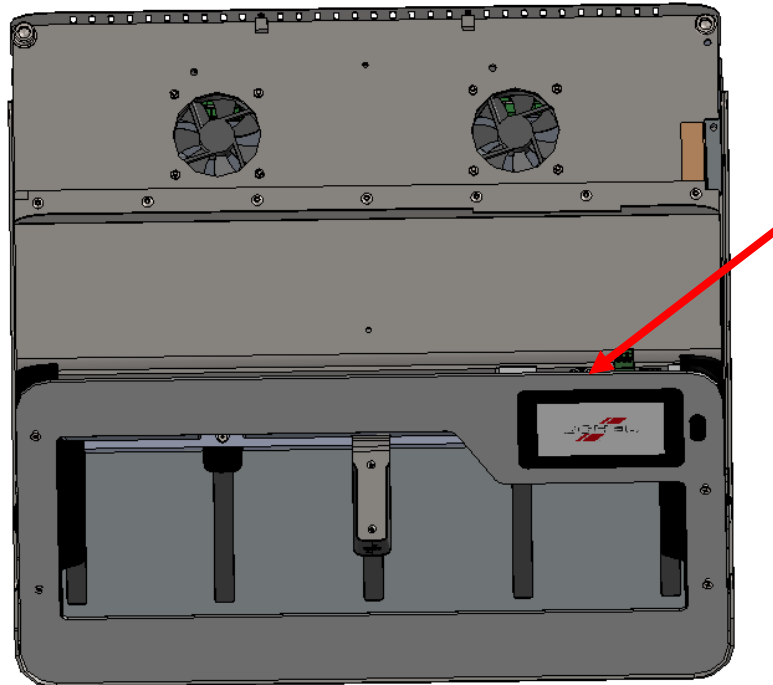


CMOS battery replacement

ERROR 007 Die The battery voltage is below 2,8 V → Replace the CR2032 button cell

6

Slide on the upper part of the housing and screw it to the side with the 6 screws. Then press the ribbon cable onto the double-sided adhesive tape on the upper part of the housing. This prevents the ribbon cable from becoming entangled with the reflector plate and, in the worst case, from being pulled off.

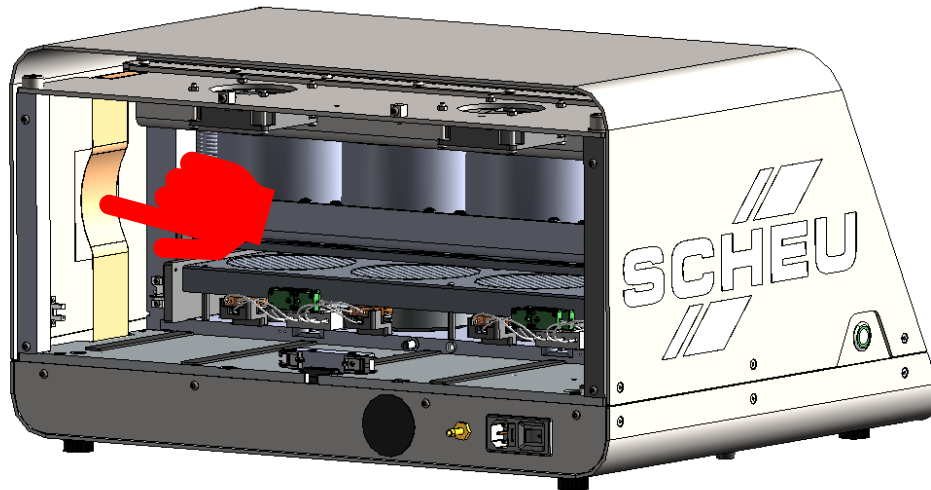

7

Now fasten the back panel with the 6 screws.

