




Repair manual BIOSTAR® 4-Aligner

8 Instructions for removing films
stuck to the protective screen

Instruction for removing films stuck to the protective screen


1 DISCONNECT THE DEVICE FROM THE POWER SUPPLY!

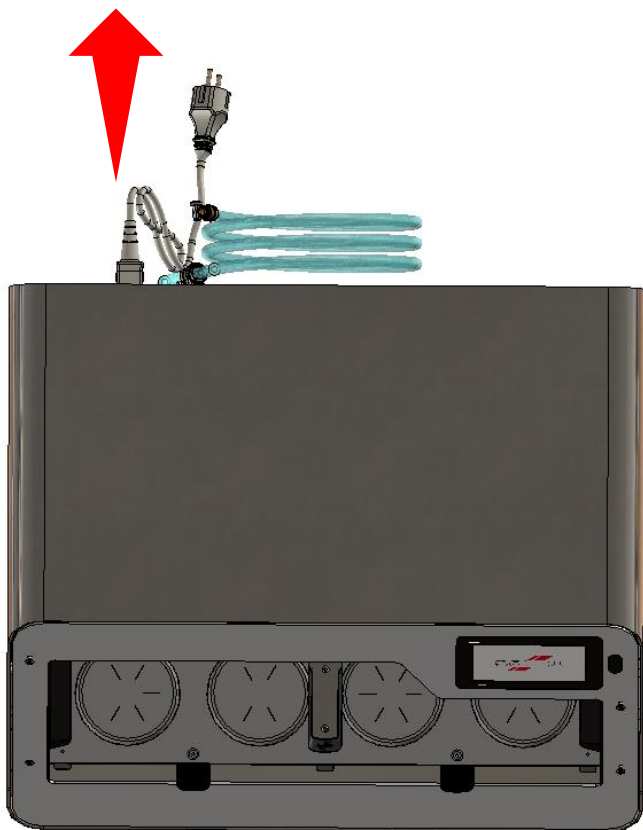

2 Turn the appliance over so that the back of the appliance is freely accessible.


Do NOT lift the device by the FRONT panel but reach under the device!

Ensure that the current software version is installed.

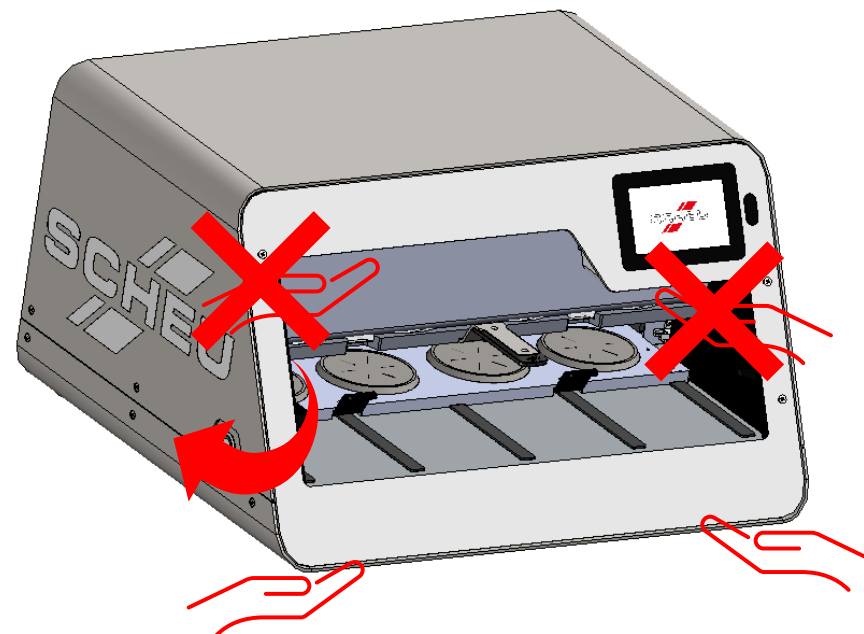


Instruction for removing films stuck to the protective screen

- 3** Loosen the 6 screws on the back panel and put the back panel to one side.

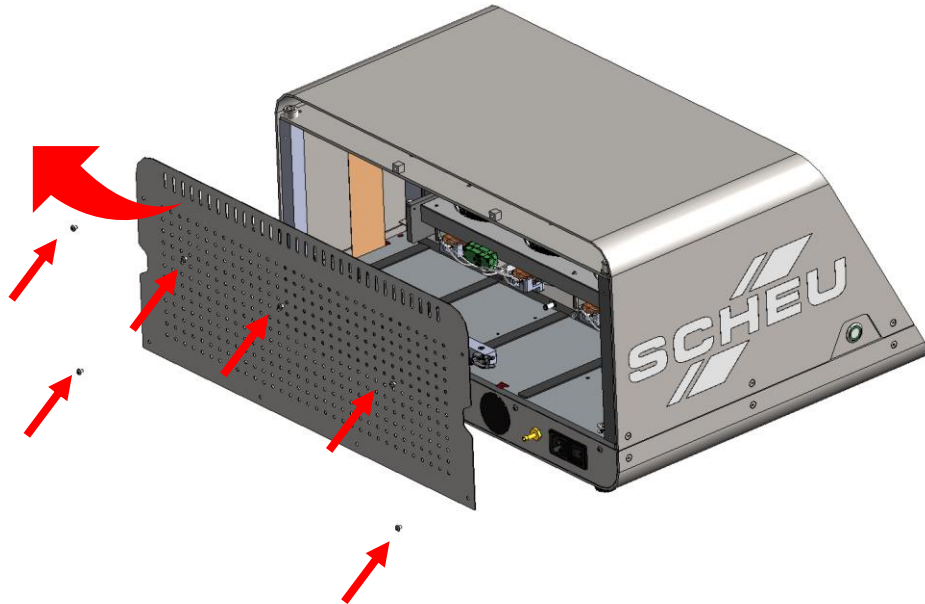


- 4** Pull the base plate with the heater back as far as it will go.

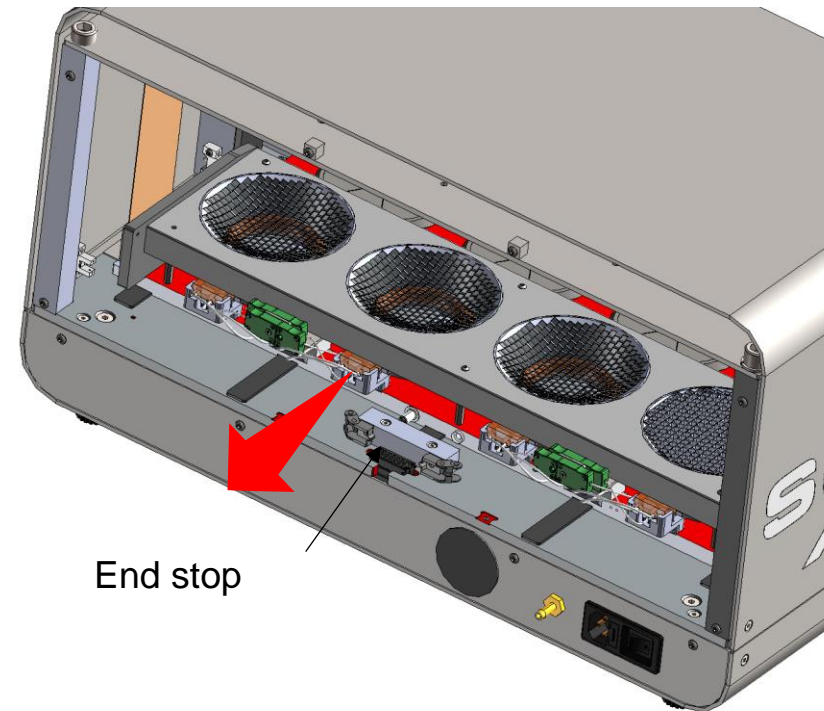


Instruction for removing films stuck to the protective screen

- 5** Loosen the 6 screws of the heater protection screen and remove it.


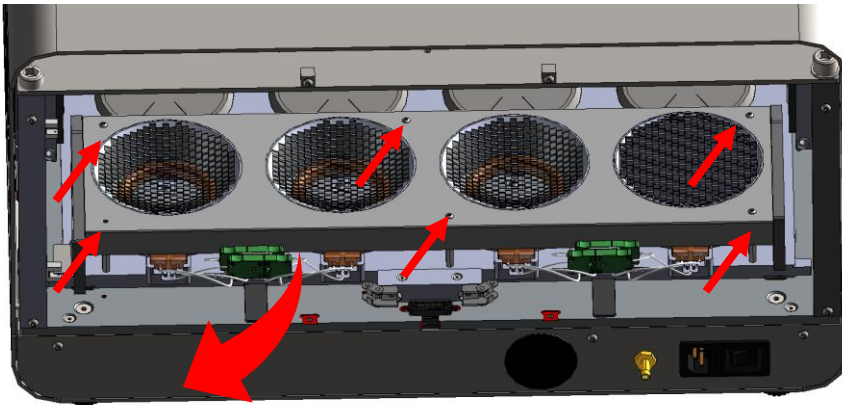


- 6** Remove the plastic residue from the protective sieve
TIP! Heat the grid at the affected area with a hot air dryer and scrape it off with an instrument.

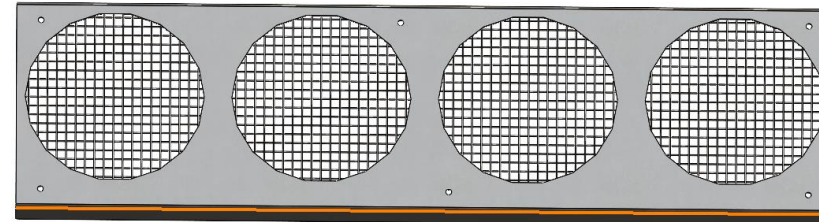

Instruction for removing films stuck to the protective screen

- 7** Fit the protective screen for the heater and screw on the rear panel. Make sure that the long side of the protective screen is facing the back panel when inserting it.
- 8** Perform the heater test in the software program to check whether the installation has been carried out successfully (see page 2).

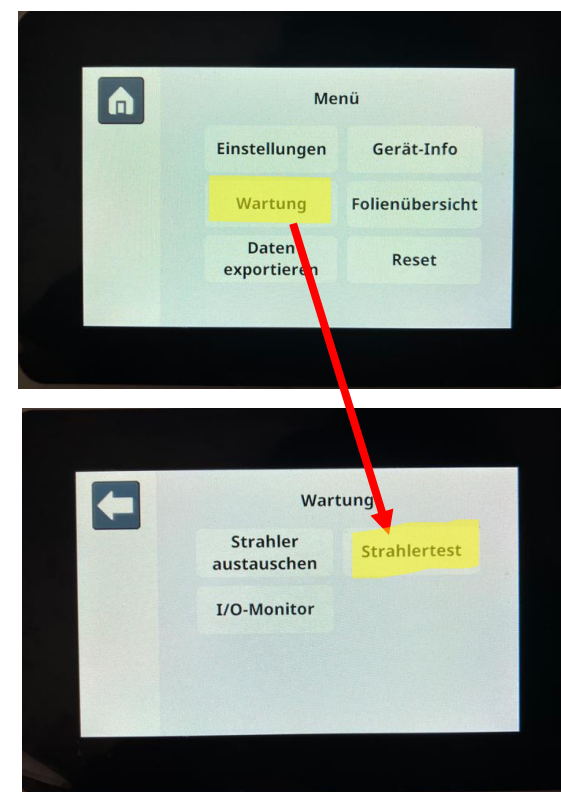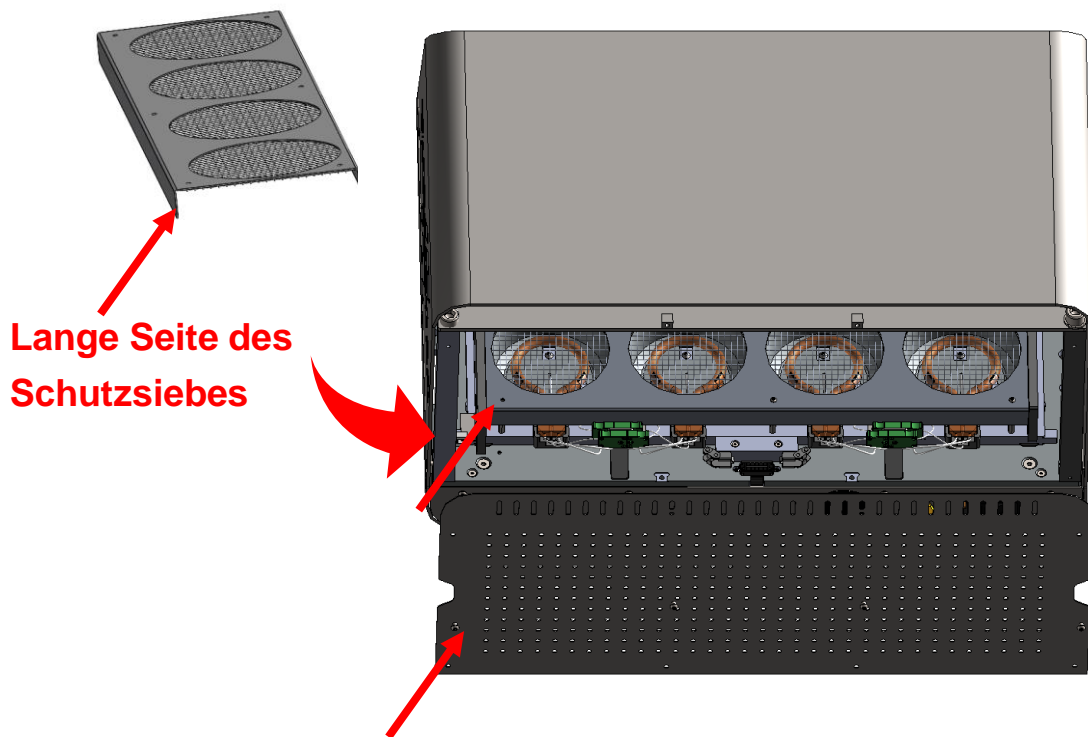

Instruction for removing films stuck to the protective screen

