




Repair manual BIOSTAR® 4-Aligner

10 Replacing the O-ring 3030.507


Replacing the O-ring 3030.507

1 DISCONNECT THE DEVICE FROM THE POWER SUPPLY!

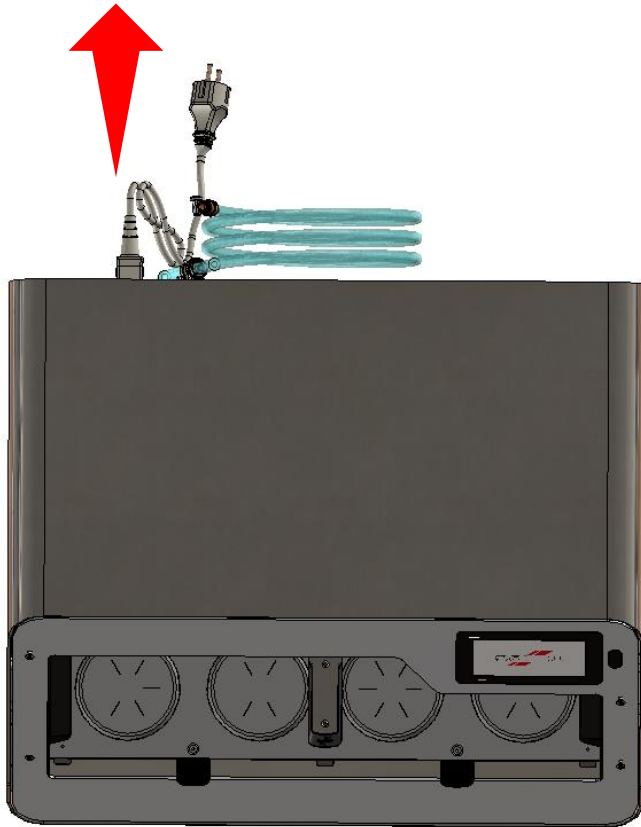

2 When moving or turning the device, do not hold the front panel, but rather the underside of the device.

Ensure that the latest software version is installed.

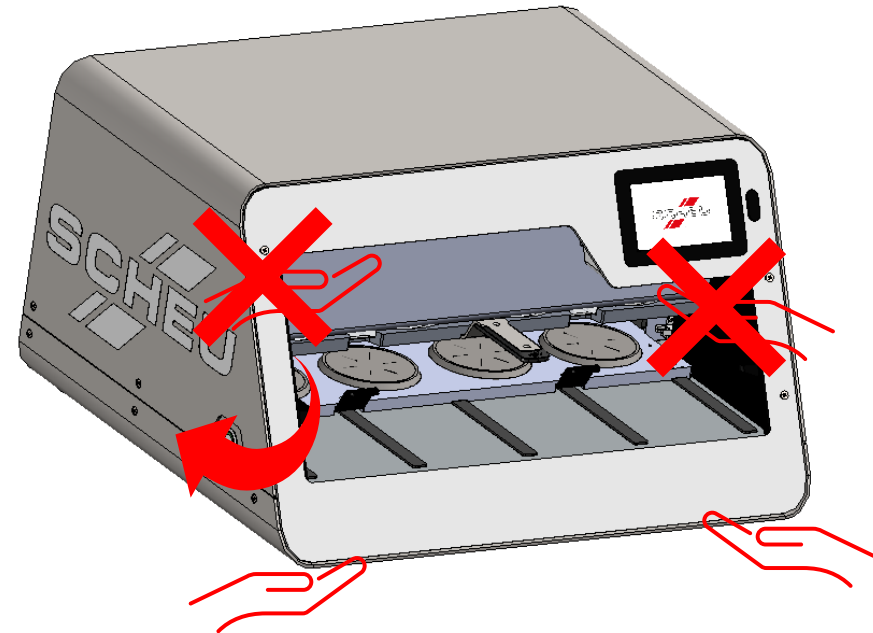

Replacing the O-ring 3030.507

3

Carefully slide a small screwdriver between the pressure chamber and the O-ring and lift the O-ring so that you can pull it out. Clean the O-ring groove and thread in a new O-ring. It must sink evenly into the groove.

