IMPRIMO® LC Cast

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Manufactured by:

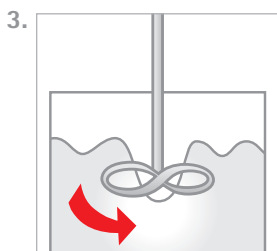
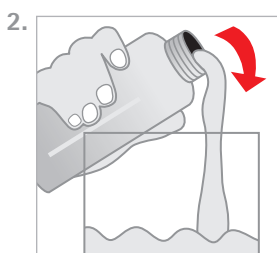
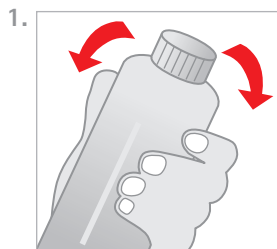
pro3dure medical GmbH
Am Burgberg 13
D-58642 Iserlohn, Germany
Phone: +49 2374 920050-0
info@pro3dure.com
www.pro3dure.com

Made in Germany

Manufactured for & distributed by:

SCHEU-DENTAL GmbH
Am Burgberg 20
58642 Iserlohn, Germany
Phone: +49 2374 9288-0
Fax: +49 2374 9288-90
service@scheu-dental.com
www.scheu-dental.com

1. Produktbeschreibung

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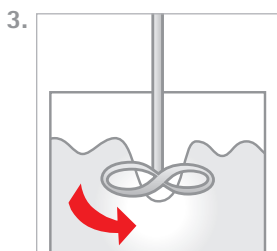
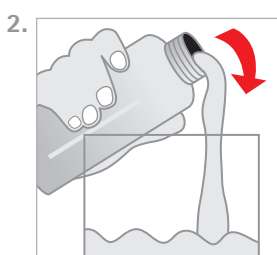
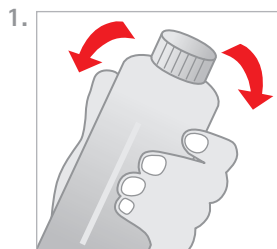
1. Product description

The generative resin **IMPRIMO® LC Cast** is a resin for the generative production of dental castforms based on image projection systems (≤ 405 nm). The formulation of **IMPRIMO® LC Cast** is optimized for the requirements of a robust production guaranteeing constant high quality. The material can be used for build processes with layer thicknesses from 10 up to 100 μm . It is recommended to use the pro3dure medical curing device **CD-1** or **CD-2** for post curing.

2. Processing

- **IMPRIMO® LC Cast** bottles should be well shaken before use (Fig. 1).
- Make sure that **IMPRIMO® LC Cast** material is temperature adjusted up to 23 °C to 30 °C.
- Carefully pour **IMPRIMO® LC Cast** into the vat of the image projection unit (Fig. 2).
- Bubbles can be removed with a cleaned spatula or by a recoater routine.
- If it is possible, always store a bottle **IMPRIMO® LC Cast** in your production unit in order to avoid temperature differences during refilling.
- For the build parameter adjustment please refer to the machine data sheet.
- After the build process is finished a direct post treatment is recommended. If this cannot be guaranteed leave the produced objects in the liquid **IMPRIMO® LC Cast** resin.
- After cleaning of the parts with isopropanol $\geq 97\%$ (approx. 3-5 min. in an ultrasonic bath) the objects are postcured in an adequate light curing unit (e.g. pro3dure's **CD-1** or **CD-2** for a period of 3-4 min.) in a protective gas atmosphere.
- The dental objects generated out of the generative resin **IMPRIMO® LC Cast** can be repaired and used with dental investment compounds as usual.
- Impurity due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin it is possible to filtrate the **IMPRIMO® LC Cast**. It is recommended to filtrate and stir up the resin on a regular base (Fig. 3). To avoid bubbles let **IMPRIMO® LC Cast** rest for 1 hour before usage.

Contains: Urethanedimethacrylates, initiators, dyes and stabilisers.

3. Important

- To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the **IMPRIMO® LC Cast** material.
- Ensure personal protective gear during processing.
- **Caution:** Polymerised resins are chemically resistant – avoid stains on clothing!
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water, consulting a doctor if necessary.
- The lot number and the best before date are indicated on each **IMPRIMO® LC Cast** packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date.

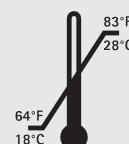
4. Safety advice

SCHEU-DENTAL GmbH is not liable for any damages caused by improper application of the material. To be used by trained specialist personnel for the purpose indicated only.

Product description:
photopolymerizable resin for the production of dental cast forms by image projection systems (≤ 405 nm)

Technical data:

- Color: red
- Density: ca. 1.1 g/ml
- Viscosity: ca. 0,3 Pa s
- Green flex modulus: Elastic modulus: ca. 800 MPa
Flexural strength: ca. 50 MPa
Elongation at break: ca. 18 %
- Post cured material: (depends on postcuring unit)
Elastic modulus: ca. 1600 MPa
Flexural strength: ca. 80 MPa
Elongation at break: ca. 5 %
Hardness: ca. 85 Shore D
- Storage:

Ordering information:

Standard packing:

1kg bottle, red
item no.: REF: 6507

These data result from measurements of a representative sample, which were determined within the scope of our quality assurance.