# IMPRIMO® LC Gingiva

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### Manufactured by:

pro3dure medical GmbH  
Am Burgberg 13  
D-58642 Iserlohn, Germany  
Phone: +49 2374 920050-0

info@pro3dure.com  
www.pro3dure.com

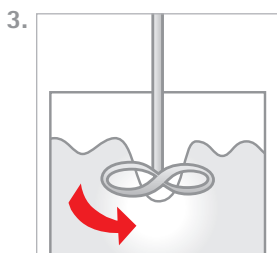
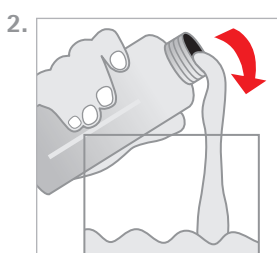
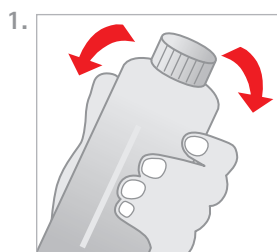
Made in Germany

### Manufactured for & distributed by:

SCHEU-DENTAL GmbH  
Am Burgberg 20  
58642 Iserlohn, Germany  
Phone: +49 2374 9288-0  
Fax: +49 2374 9288-90

service@scheu-dental.com  
www.scheu-dental.com

## 1. Produktbeschreibung

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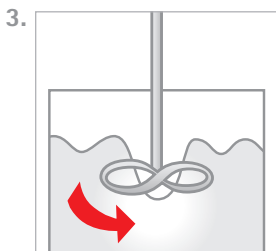
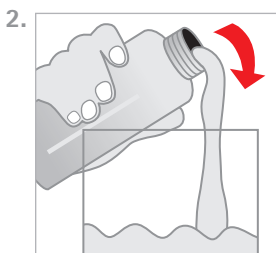
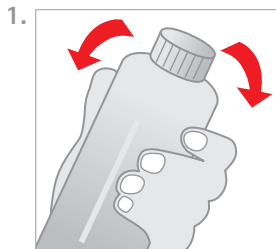
## 1. Product description

The generative resin **IMPRIMO® LC Gingiva** is a resin for the generative production of dental objects based on image projection systems ( $\leq 405$  nm). The formulation of **IMPRIMO® LC Gingiva** is optimized for the requirements of a robust production guaranteeing constant high quality. The material can be used for build processes with layer thicknesses from 25–100  $\mu\text{m}$ . It is recommended to use the **CD-1** or **CD-2** curing device from pro3dure medical for post curing.

## 2. Processing

- **IMPRIMO® LC Gingiva** bottles should be well shaken before use (Fig. 1).
- Make sure that **IMPRIMO® LC Gingiva** material is adjusted to temperature range 23 °C to 30 °C.
- Carefully pour **IMPRIMO® LC Gingiva** into the vat of the image projection unit (Fig. 2).
- Bubbles can be removed with a cleaned spatula or by a recoater routine.
- If it is possible, always store a bottle **IMPRIMO® LC Gingiva** in your production unit in order to avoid temperature differences during refilling.
- For the build parameter adjustment please refer to the machine data sheet.
- After the build process is finished a direct post treatment is recommended. If this cannot be done at the time leave the produced objects in the liquid **IMPRIMO® LC Gingiva** resin.
- After cleaning the parts with isopropanol  $\geq 97\%$  (approx. 3–5 min. in an ultrasonic bath) the objects are postcured in a light curing unit (e.g. pro3dure's **CD-1** or **CD-2** for a period of 3–4 min.) in a protective gas atmosphere.
- The dental objects generated out of the generative resin **IMPRIMO® LC Gingiva** can be repaired with **IMPRIMO® LC Gingiva** resin.
- Impurity due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin it is possible to filtrate the **IMPRIMO® LC Gingiva**. It is recommended to filter and stir up the resin on a regular base (Fig. 3). To avoid bubbles let **IMPRIMO® LC Gingiva** rest for 1 hour before usage.

**Contains:** Urethanedimethacrylates, initiators, dyes and stabilisers.

## 3. Important

- To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the **IMPRIMO® LC Gingiva** material.
- Wear personal protective gear during processing.
- **Caution:** Polymerised resins are chemically resistant – avoid stains on clothing!
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water. Consult a doctor if necessary.
- The lot number and the expiration date are indicated on each bottle of **IMPRIMO® LC Gingiva**. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date.

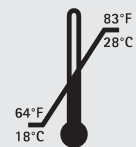
## 4. Safety advice

SCHEU-DENTAL GmbH is not liable for any damages caused by improper application of the material. To be used by trained specialist personnel for the purpose indicated only.

**Product description:**  
photopolymerizable resin for the production of dental objects by image projection systems ( $\leq 405$  nm)

### Technical data:

- Color: rose
- Density: ca. 1.1 g/ml
- Viscosity: ca. 2,5 Pa s
- Post cured material: (depends on postcuring unit)  
Hardness: ca. 60 Shore A
- Storage:

### Ordering information:

**Standard packing:**

1kg bottle,  
rose  
item no.: REF 6509

These data result from measurements of a representative sample, which were determined within the scope of our quality assurance.