IMPRIMO® LC Model stone

GB 1. Intended use / Indication

Light-curable polymerizable resin intended to be used in conjunction with extra-oral curing light equipment for the fabrication, by additive manufacturing, of dental appliances. Indicated for the production of dental models.

2. Contraindication

IMPRIMO® LC Model stone is contraindicated for any use on patients (**IMPRIMO® LC Model stone** is only a laboratory product).

3. Intended Users

Dentist, dental technicians

4. Requirements

Software - information available from:

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Hardware (3D Printing) - information available from:

ASIGA · 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) - information available from:

NK Optik GmbH (otoflash G171) · Isarstr. 2, D-82065 Baierbrunn, Germany

Additional specifications on www.scheu-dental.com.

5. Material

IMPRIMO® LC Model stone consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

6. Geometric presetting

According to the design specifications of the applicable CAD software.

7. Material parameters

Radiation penetration depth controllable by exposure time:

50 µm

100 µm

Instructions for use
Gebrauchsanweisung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso

8. Manufacturing process (fig. 1-10)

1. Prepare data (CAD & build preparation).
2. Choose process parameters (build style, etc.).
3. Transfer prepared data to 3D printing device.
4. Prepare 3D printing – shake bottle.
5. Fill resin tank of 3D printing device.
6. Build the parts.
7. Clean parts (with IPA ≥ 97 % or equivalent cleaning agent) approx. 4 min. in an ultrasonic bath or equivalent device – precleaning recommended.
8. Dry parts, until there are no residues of IPA or equivalent cleaning agent.
9. Post curing (2 x 2000 flashes – otoflash G171): inert atmosphere recommended.
10. Finish parts.

9. Finishing processes

According to the further uses (e.g. insulating, blocking out, etc.).


10. Notice

Please follow the instructions provided by the software manufacturer in terms of parameter settings and design recommendations. Please follow the instructions provided by the hardware manufacturer in terms of parameter settings/printing & post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. **Caution:** The lot number and the best before date are indicated on each packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Do not breathe dust/fume/gas/mist/vapours/spray. Do not eat, drink or smoke when using this product. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Call a POISON CENTER/doctor if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Get medical advice/attention. Get medical advice/attention if you feel unwell. Keep container tightly closed. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Wash contaminated clothing before reuse. Collect spillage. Store locked up. Protect from sunlight. Store in a well-ventilated place. Dispose of contents/container according to official regulations.

11. Hazard statements

May cause an allergic skin reaction. Suspected of damaging the unborn child. Causes damage to organs through prolonged or repeated exposure. Toxic to aquatic life with long lasting effects.

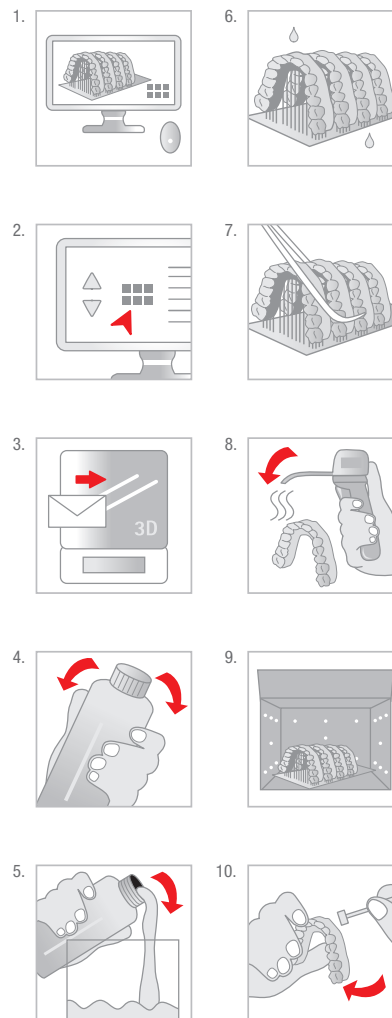UMDNS 16-697

Physical properties */
Physikalische Eigenschaften */
Propriétés physiques */
Propiedades físicas */
Proprietà fisiche *:

IMPRIMO® LC Model stone

■ Ultimate flexural strength/
Biegefestigkeit/
Résistance à la flexion/
Resistencia a la flexión/
Resistenza a flessione
MPa (ISO 20795-1)
≥ 100

■ Flexural modulus/
Biegemodul/
Module en flexion/
Módulo de flexión/
Modulo a flessione
MPa (ISO 20795-1)
≥ 2500

Ordering information/
Bestellinformationen/
Informations de commande/
Información sobre pedidos/
Informazioni sull'ordine:

IMPRIMO® LC Model stone

1 kg:
λ ≤ 405 nm

beige/
beige/
beige/
beige/
beige/
beige/
REF: 6566

ivory/
ivory/
ivory/
ivory/
ivory/
ivory/
REF: 6567

grey/
grau/
gris/
gris/
grigio
grigio
REF: 6568

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